
Blood Pressure Monitor BU 512

Instruction Manual Please read carefully!

GB Unit and LCD-Display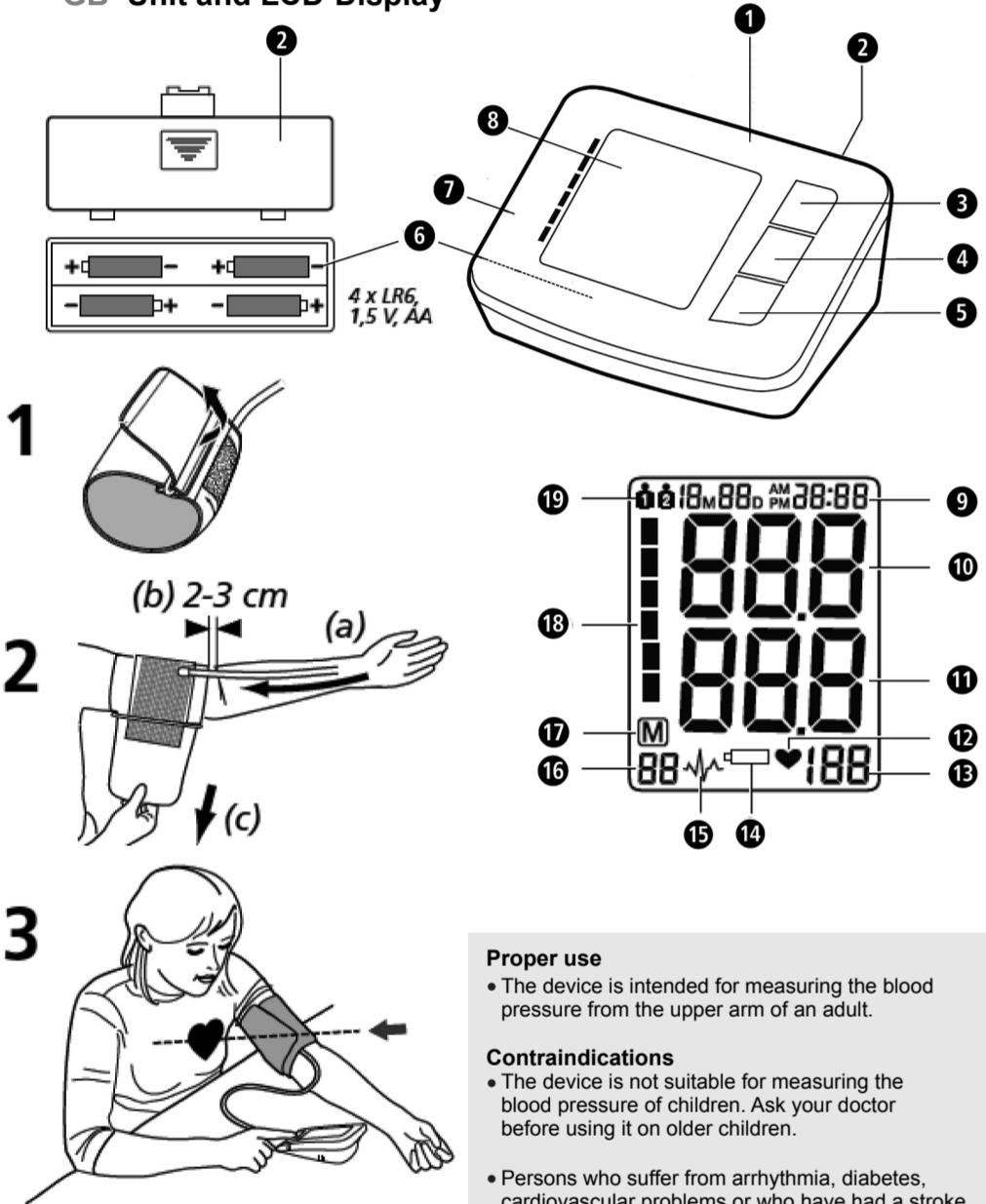

Explanation of symbols

IMPORTANT
Follow the instructions for use!
Non-observance of these instructions
can result in serious injury or damage
to the device.

WARNING
These warning notes must be observed
to prevent any injury to the user.

CAUTION
These notes must be observed to prevent
any damage to the device.

NOTE
These notes give you useful additional
information on the installation or
operation.

Device classification: type BF applied part

IP21 Protection: IP21

LOT LOT number

Manufacturer

Date of manufacture

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.
- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- Consult your physician to get more information about your blood pressure. A self-diagnosis and self-treatment based on the measured values may be dangerous. Always follow the instructions given by your physician!
- If you experience discomfort during a measurement, such as pain in the upper arm or other complaints, press the START/STOP-button ④ to release the air immediately from the cuff. Loosen the cuff and remove it from your arm.
- The device only works correctly with the fitting cuff.
- The unit is not suitable for children.
- Do not allow children to use the unit. Medical products are not toys!
- Please ensure that the unit is kept away from the reach of children.
- Do not put the air tube around your neck - danger of suffocation!
- The swallowing of small parts like packaging bag, battery, battery cover and so on may cause suffocation.
- The user needs to check that the equipment functions properly and that it is in proper working condition before being used.
- Only the cuff provided can be used, and cannot be changed to any other branded cuff. It can only be replaced with a cuff of exactly the same type.
- The device may not be operated in rooms with high levels of radiation or in the vicinity of high-radiation devices, such as radio transmitters, mobile telephones or microwaves, as this could cause functional defects or incorrect measurements.
- Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquids (such as alcohol) are present.
- No modification of this unit/device is allowed.
- Please do not attempt to repair the unit yourself in the event of malfunctions, since this will void all warranty claims. Only have repairs carried out by authorised service centres.
- Protect the unit against moisture. If liquid does manage to penetrate into the unit, remove the batteries immediately and do not continue to use the unit. In this case contact your specialist dealer or notify us directly.
- Please do not use a dilution agent, alcohol or petrol to clean the unit.
- Please do not hit the device heavily or let it fall down from a high place.
- Do not use mobile phones in the range of the device. This may lead to malfunctions.
- If the unit is not going to be used for a long period, please remove the batteries.

Proper use

- The device is intended for measuring the blood pressure from the upper arm of an adult.

Contraindications

- The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- Persons who suffer from arrhythmia, diabetes, cardiovascular problems or who have had a stroke should consult their doctor before using the device.

- Do not disassemble batteries!
- Replace all batteries if the unit display shows the low battery symbol.
- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed, seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period of 3 months or more!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw batteries into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Unit and LCD display

- | | |
|------------------------------------------------------------|--------------------------------------|
| ① Blood Pressure Monitor BU 512 | ② Cover for Battery Compartment |
| ③ MEM button (Memory Recall) | ④ START/STOP button |
| ⑤ SET button | ⑥ Battery Compartment (on underside) |
| ⑦ Push-in Connector for air tube | ⑧ LCD screen (display) |
| ⑨ Display of Date/Time | ⑩ Display of Systolic Pressure |
| ⑪ Display of Diastolic Pressure | ⑫ Pulse symbol |
| ⑬ Display of Pulse Rate | ⑭ Pulse symbol |
| ⑮ Irregular heartbeat display | ⑯ Change Battery symbol |
| ⑰ Memory-Symbol | ⑯ Memory Location number |
| ⑱ Blood Pressure Indicator (green - yellow - orange - red) | |
| ⑲ User memory 1 / 2 | |

Scope of supply

- Please check first of all that the unit is complete. The following parts are included as standard:
- 1 medisana blood pressure monitor BU 512
 - 1 cuff with air tube
 - 4 batteries (type AA, LR6) 1.5 V
 - 1 storage pouch
 - 1 instruction manual

Please contact your supplier if you notice any transport damage on unpacking the unit.

What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

How is blood pressure measured?

The medisana BU 512 is a blood pressure unit which is used to measure blood pressure at the upper arm. The measurement is carried out by a microprocessor, which, via a pressure sensor, measures the vibrations resulting from the inflation and deflation of the cuff over the artery.

Blood pressure classification

Blood pressure classification without making any allowance for age.

Classification	systolic (mmHg)	diastolic (mmHg)	Display area ⑬
Stage 3 Hypertension (emergency)	≥180	≥110	Red
Stage 2 Hypertension (moderate)	160 - 179	100 - 109	Orange
Stage 1 Hypertension (mild)	140 - 159	90 - 99	Yellow
High - Normal	130 - 139	85 - 89	Green
Normal	120 - 129	80 - 84	Green
Optimal	<120	<80	Green

WARNING
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Starting up

Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment ⑥ is located on the underside of the unit. Open it by slightly pushing it and moving it upwards. Remove it and insert the 4 AA LR6 type 1.5 V batteries supplied. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment). Close the battery compartment. Replace the batteries if the change battery symbol ⑯ appears on the display ⑩ or if nothing appears on the display after the unit has been switched on.

Settings

1. User setting:

Press SET button ⑤ when power off, ① or ② appear on the display. By pressing MEM button ③ you may choose between user ① or ②.

Press SET button ⑤ to confirm the user setting. Afterwards, you can proceed with the year setting mode.

2. Year setting:

The year figure starts to flash for setting. Keep pressing the MEM button ③ until the actual year figure appears. Press SET button ⑤ to confirm the year setting. Afterwards, you can proceed with the month/date setting mode.

3. Month / date setting:

The month figure starts to flash for setting. Keep pressing the MEM button ③ until the actual month figure appears. Press SET button ⑤ to confirm the month setting. Afterwards, the day needs to be set in the same way. Afterwards, you can proceed with the time setting mode.

4. Time setting:

The hour figure starts to flash for setting. Keep pressing the MEM button ③ until the actual hour figure appears. Press SET button ⑤ to confirm the hour setting and proceed in the same way for minute setting. Conclusively, CL appears in the display. You may delete stored values at this point - please check chapter "Memory- clear of measurements". Press SET button ⑤ to leave the setting mode. If the batteries are changed, the set information is lost and needs to be input again.

Troubleshooting

Problem	Check	Cause and solutions
No power	Check the battery power and if the polarity is correct.	Replace all batteries and pay attention to correct polarity during insertion.
No inflation	Check for proper hose condition and if it is properly connected to the device.	Connect the air hose properly. Use a new cuff.
Err appears and the measurement is aborted	Check whether you have moved the arm during inflation or if you have talked during measurement.	Keep your arm and your body still. Do not talk during measurements.
Leak cuff	Check whether the cuff is too loose or damaged. Close it tightly. Use a new cuff.	Put on the cuff correctly and

Please contact the distributor resp. the service centre if you can't solve the problem. Do not disassemble the unit by yourself.

Taking a blood pressure measurement

After the cuff has been appropriately positioned, the measurement can begin:

- Switch the unit on by pressing the START/STOP button ④.
- If the START/STOP button ④ is pressed, all display characters are shown (display test). This test can be used to check that the display is indicating properly and in full.
- The unit is ready for measurement. The number 0 flashes for 2 seconds and two short beeps are emitted. The device then automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display.
- The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol ⑩ on the display starts to flash, you can hear the beep for every heartbeat once the heartbeat signal is detected.
- When the measurement is finished, you can hear a long beep and cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value appear on the display ⑩. The blood pressure indicator ⑯ is displayed next to the relevant coloured bar depending on the blood pressure classification. If the unit has detected an irregular heartbeat, the irregular heartbeat display ⑯ also appears.

WARNING
Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor.

- The readings are automatically saved in the previously selected memory ① or ②. Up to 90 measured values with date and time can be stored in each memory.
- The measurement readings remain on the display. The unit switches off automatically after approx. 180 seconds if no buttons are pressed. The unit may also be switched off using the START/STOP button ④.

Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell) the START/STOP button ④ can be pressed at any time. The device immediately decreases the cuff pressure automatically.

Display stored values

This unit features 2 separate memories, each with a capacity of 90 memory slots. Results are automatically stored in the memory selected by the user. Press the MEM button ③ when power off, to call up the measured values stored. All values average appear on the display. Press the MEM button ③ again to call up the last measurement value. Repeatedly pressing the MEM button ③ displays the respective values measured previously. If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 8 seconds in memory recall mode. Memory recall mode can be exited at any time by pressing the START/STOP button ④, which will also switch the unit off. The oldest value is deleted if 90 measured values have been stored in the memory and a new value is saved.

Memory - clear of measurements

If you are sure that you want to permanently remove all stored memories, press the SET button ⑤ for 7 times until CL appears when power off. Press the START/STOP button ④. CL will flash for 3 times to clear all the memories. After this press MEM button ③. M and "no" will be shown on the display which mean that no memory is stored.

Error messages and error remedying

Error indicators

The following symbol will appear on the display when measuring abnormal:

Symbol	Cause	Correction
E-1	Weak signal or sudden pressure change	Put on the cuff correctly. Repeat the measurement in the correct way.
E-2	External strong disturbance	When near cell phone or other high radiant device, the measurement may fail. During measurement, do not move and do not speak.
E-3	Error during inflation process	Put on the cuff correctly. Make sure, that the hose is properly connected. Repeat the measurement.
E-5	Abnormal blood pressure	Repeat the measurement after relaxing 30 minutes. If you get unusual readings for 3 times, please contact your doctor.
Weak batteries	The batteries are too low or empty. Replace all 4 batteries with new 1.5 V type AA LR6 batteries.	

Cleaning and care

Remove the batteries before cleaning the unit. Clean the unit using a soft cloth lightly moistened with a mild soapy solution. Never use abrasive cleaning agents, alcohol, naphta, thinner or gasoline etc. Never immerse the unit in any component in water. Be cautious not to get any moisture in the main unit. Do not wet the cuff or attempt to clean the cuff with water. Using a dry cloth, gently wipe away any excess moisture that may remain on the cuff. Lay the cuff flat in an unrolled position and allow the cuff to air dry. Do not expose the unit to direct sunlight; protect it against dirt and moisture. Do not expose the unit to extreme hot or cold temperatures. Keep the device in the storage case when not in use. Store the unit in a clean and dry location.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

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