


medisana®

Digital Personal Scale PS 200

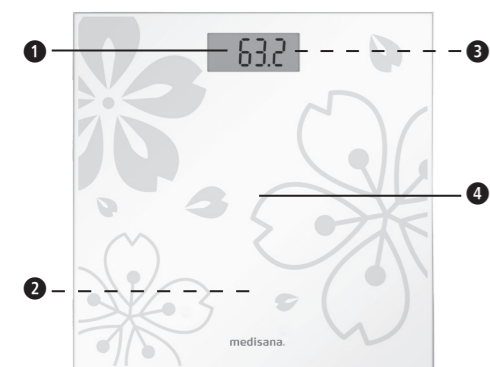

GB Unit and controls

- 1 Display
- 2 Battery Compartment (on underside)
- 3 Unit key (on underside) for kilograms (kg) pounds (lb) stones (st)
- 4 Standing area

Legend:



This instruction manual belongs to this device. The instruction manual includes information on the initial start-up and handling. Read this instruction manual completely. Failure to follow these instructions may result in serious injury or damage to the equipment.



WARNING!

These warnings must be followed to prevent possible injury to the user.



LOT number



Manufacturer



48845_PS 200_manual_GB_202206_L1

GB Safety instructions



Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- Any treatment or diet for people who are underweight or overweight requires the qualified advice of a professional (doctor, dietician).
- Risk of tipping over! Do not place the scale on an uneven surface. Never step onto a corner of the scales. When weighing yourself, do not stand on one side or the edge of the scales.
- Risk of slipping! Do not stand on the scale with wet feet. Do not stand on the scale wearing socks.
- Use the device only according to its intended purpose as specified in the instruction manual. The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The scale has been manufactured for domestic use. It is not suitable for commercial use in hospitals or other medical institutions.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- The scales have a measuring range up to 180 kg, 396 lb / 28 st 4 lb. Do not overload the scales.
- Place the scales on a firm, level base. Soft, uneven surfaces are unsuitable for the measuring process and lead to incorrect results.
- Do not use the scale in a location where extremely high temperatures or humidity may occur.
- Keep the scale away from water.
- Handle the scale with care. The scale must not be subjected to impacts or vibration. Do not drop the scale.
- Stand carefully on the scale. Do not jump or hop onto the standing area. This could lead to defects of the weighing mechanism.
- Do not attempt to dismantle the scale, otherwise the warranty will be invalidated. This unit contains no user-maintainable or user-replaceable parts.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.

Battery safety instructions

- Do not disassemble batteries!
- Remove discharged batteries from the device immediately! Increased risk of leakage!
- Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the batteries correctly, observing the polarity!
- Always keep the battery compartment well closed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep the batteries out of the reach of children!
- Do not re-charge the batteries! **There is a risk of explosion!**
- Do not short-circuit the device! **There is a risk of explosion!**
- Do not throw into a fire! **There is a risk of explosion!**
- Store unused batteries in the packaging and not near metal objects to avoid a short circuit!

Scope of delivery

First, check whether the device is complete and is not damaged.

If in doubt, do not operate the device and contact your service center.

The scope of delivery includes:

- 1 **medisana** Digital Personal Scale **PS 200**
- 1 Instruction manual



Packaging is recyclable or can be recycled into raw materials. Please dispose of unwanted packaging material properly. If you notice any transport damage while unpacking, please contact your dealer immediately.



WARNING

Make sure that children do not play with the packaging foils. Risk of suffocation!

Requirements for correct measurement results

- Always weigh yourself in the same conditions.
- Measure at the same time of day.
- Stand upright and calm.
- It is best to take the measurement in the morning, after showering or bathing, with dry foot.

Insert/change batteries

Before putting your digital scale into operation, insert one 3V lithium cell CR2032 battery into the device. Open the battery compartment 2 on the underside of the device and insert the batteries. Pay attention to the polarity (illustration in the battery compartment). Replace the battery compartment cover and push it in so that it clicks into place. Change the batteries if the battery change symbol "Lo" appears on the display 1 or if nothing is shown on the display after the unit has been operated.

Initialise scale

Make sure that the scale is on firm and level ground. If you want to weigh yourself, but have moved the scale immediately before, you must first initialise the scale. Briefly press the middle of the scale surface with your foot. The display will show "0.0". When the scale has turned off, it is ready for the "step-on" function. If the scale has not been moved beforehand, this process is omitted.

Simple weighing

1. Stand on the scale and stand still. The device switches on automatically and the display 1 briefly shows 0.0kg.
2. Your weight will be measured, flashes twice and is then finally displayed.
3. Step down off the scale. The scale switches off automatically after approx. 10 seconds.

Change the unit of weight

Press your foot on the middle of the surface of the scales shortly to activate the device. Then press button 3 until the unit you wish is shown.

Error messages

Err = The scale is overloaded. **Lo** = The battery is weak and needs to be replaced.

If your scale does not work as desired, check the following before contacting the service center

- Check that the battery has been correctly inserted
- Check that you have selected the correct weight unit.
- Check whether the scale is standing freely on a firm and level surface. It must not touch a wall or another object.
- Repeat the weighing process.

Cleaning and care

Never use aggressive cleaning agents or stiff brushes. Clean the scale only with a soft, slightly damp cloth. Never use harsh detergents or alcohol. No water should enter the device. Only use the device again when it is completely dry.

Instructions for disposal



This device must not be disposed of with domestic waste. Metals are reusable or can be recycled into raw materials. Dispose of a device that is no longer needed at a collection point in your town/city so that it can be disposed of in an environmentally friendly way. Please contact your local authority or retailer with regard to disposal procedures. Remove the battery before you dispose of the device. Do not dispose of spent batteries in domestic waste, rather in special waste or at a battery collection point at your retailer!

Technical specifications

Name and model no.	: medisana Digital Personal Scale PS 200
Voltage supply	: 3V lithium cell CR2032
Measuring range	: up to 180 kg / 396 lb / 28 st 4 lb
Graduation	: 0.1 kg / 0.2 lb / ¼ lb
Dimensions	: approx. 280 x 280 x 20 mm
Weight	: approx. 1.3 kg
Article no.	: 48845
EAN no.	: 40155 88 48845 0

In the course of constant product improvements, we reserve the right to make technical and design changes.

Warranty and repair conditions

In case of warranty please contact your specialist shop or the service centre directly. If you need to return the device, please indicate the defect and enclose a copy of the purchase receipt.

The following warranty conditions apply:

1. All **medisana** products are guaranteed for one year from the date of purchase. The date of purchase is to be proven in case of warranty by the purchase receipt or invoice.
2. Defects due to material or manufacturing defects shall be eliminated free of charge within the warranty period.
3. A warranty service does not extend the warranty period for either the equipment or any replacement parts.
4. The following are excluded from the guarantee:
 - a. any damage caused by improper handling, e.g. by non-observance of the instruction manual.
 - b. Damage due to repair or intervention by the purchaser or unauthorised third parties.
 - c. Transport damage that has occurred on the way from the manufacturer to the consumer or when sending it to the service centre.
5. Liability for direct or indirect consequential damages caused by the device is also excluded if the damage to the device is recognised as a warranty case.

medisana®

Unit E&F, 31/F, YHC Tower, 1 Sheung Yuet Road, Kowloon Bay, Kowloon, Hong Kong