
medisana®

Clinical Infrared Thermometer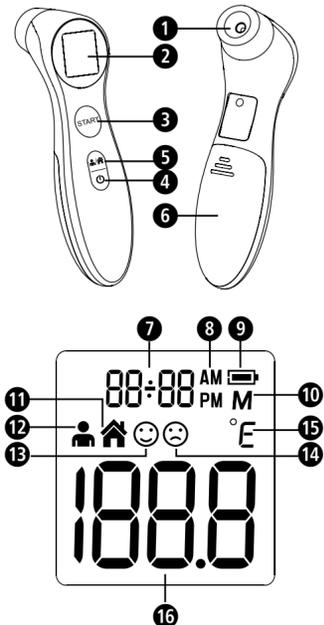

Non-contact measurement

GB Unit and controls

- ➊ Sensor
- ➋ Power button
- ➌ Time and date display
- ➍ Memory display
- ➎ Normal temperature indicator
- ➏ Measurement result
- ➐ Display
- ➑ Mode button
- ➒ AM/PM (12-hour clock)
- ➓ Forehead mode
- ➔ Fever indicator
- ➕ START button
- ➖ Battery compartment
- ➗ Battery symbol
- ➘ Object mode
- ➙ Measurement unit (C or F)

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

Classification: Type BF applied part

Temperature range

Keep dry

IP22

Protection classification against solid objects and water

LOT number

Manufacturer

Date of manufacture

GB Infrared Multi Functional Thermometer

Manual - Please read carefully!

Safety Information

IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!
Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

INTENDED USE:

Thank you for purchasing our Infrared Thermometer. This Infrared Thermometer is a non-sterile, reusable, handheld device. It can be used by consumers in home care environment and doctors in clinic as reference. It is intended for measuring human body temperature of three months and above of people by detecting infrared heat from the center of the forehead.

- Only use the unit for the specific purpose described in the instruction manual. Any misuse will void the warranty.
- The unit is not designed for commercial use.
- Using this thermometer is no substitute for consulting a doctor.
- The thermometer and the patient must remain in a stable environment for at least 30 minutes before measuring the temperature.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children must be supervised to ensure that they do not play with the device.
- The temperature in the forehead and temple area differs from the internal temperature, which is taken orally or rectally. If the measurement therefore does not match the patient's own perception or is unusually low, repeat the measurement every 15 minutes. As a reference, you can also measure the internal body temperature using a conventional oral or rectal thermometer.

- A control measurement using a conventional thermometer is recommended in the following cases:
 1. If the reading is surprisingly low
 2. For new-born infants up to 100 days old
 3. For children under three years of age who have a weakened immune system or who react unusually in the presence or absence of fever
 4. If you are using the thermometer for the first time or want to familiarise yourself with it
- If a fault occurs or the unit is damaged, do not attempt to repair the unit yourself, as this will void the warranty. Contact your dealer and only have repairs carried out by authorized service partners.
- Do not modify and / or open the device. It contains small parts which might be swallowed by children.
- Do not use the thermometer if it is damaged.
- Do not use the thermometer in damp environments. Do not immerse the thermometer in water or other liquids. **It is not watertight.**
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations.
- Keep a minimum distance of 3.3 m from such devices when using this device.
- Do not shake or tap the thermometer. Do not drop it.
- Protect the device from direct sunlight, extremely high or low temperatures, dirt and dust.
- If the thermomter is not going to be used for a long period, remove the battery.

WARNING - The measurement results obtained by this thermometer do not represent a diagnosis and do not replace the consultation of a doctor, who should always be consulted in case of doubt.

Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre. The following parts are included:

- 1 medisana Clinical Infrared Thermometer
- 2 Batteries (AAA type, LR03) 1.5V
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING - Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

Taking a temperature reading - quickly and accurately!


The medisana Clinical Infrared Thermometer allows precise, contact-free measurement of body temperature (at a distance of approximately 15-50 mm). The temperature is taken above the eyebrow between the forehead and the temple. The value of the energy radiated there is measured and converted to a temperature value. The reading is displayed after 1 second. This enables you, for example, to take babies' temperature without disturbing their sleep.

Operation - Getting started

When the battery symbol ➖ appears in the display after a longer time of usage, the batteries should be replaced soonest. When the battery symbol ➗ appears or nothing at all appears in the display ➔, the batteries need to be replaced immediately.

Removal

Remove the lid of the battery compartment ➖ and take out the batteries.

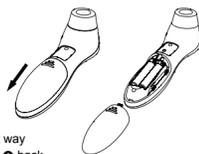Insertion

Put in two new 1.5V batteries of type LR03/AAA. Make sure the batteries are the right way around (see the diagram in the battery compartment ➖). Put the battery compartment ➖ back.

Time and Date settings

1. When using the thermometer for the first time, please set the parameters of the thermometer. When the thermometer is off, press and hold the Power button ➋ to enter into setting model. The device can display time in either 12-hour / 24-hour format. Press the START button ➕ to select the format. Press Power button ➋ to confirm and enter into time settings.
2. Press START button ➕ to adjust the hour and press Power button ➋ to confirm. Use the same method to adjust the minutes and enter to date settings.
3. Please use the same method to adjust the year, month and date settings respectively. After the date is set, press Power button ➋ to exit the setting mode.

BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the battery correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Remove the battery from the device if it is not going to be used for an extended period!
- Keep batteries out of childrens reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery

NOTES
• The device will automatically switch off if there is no input for approx. 60 seconds.

Before use

Acclimatise the thermometer to the temperature of the room: For an accurate reading of a patient's temperature, their body must first acclimatise to the temperature of the room where the measurement will be made. The same applies to the Clinical Infrared Thermometer. When taken somewhere else, it automatically adjusts itself to the new environment. The thermometer takes a certain amount of time to equalise its own temperature to that of the room. This depends on the difference between the two temperatures and can take more than 30 minutes. Precise readings can only be taken once this process is completed.

Measuring forehead temperature

- NOTES For the most accurate readings:**
- Do not take a baby's temperature during or directly after breastfeeding.
 - Patients should not drink or eat directly before or during a measurement, and they should avoid any physical exertion.
 - Clean the area where you are taking the temperature and remove any dirt, hairs or sweat before moving the sensor ➊ to the measuring position.
 - Do not remove the thermometer from the measuring position until it has beeped to signal the end of the reading.
 - If the sensor becomes dirty during or after a reading, clean it with a cotton bud dipped in cleaning alcohol before putting the thermometer away. After cleaning, allow at least 10 minutes drying time before taking temperatures.
 - Always take the temperature in the same place, because the readings will vary in different environments.

The thermometer's safety and accuracy has been inspected and confirmed, provided it is used in accordance with these instructions.

1. Press the Power button ➋ to switch on the thermometer. Then a self-test is executed in which all elements of the display ➌ are shown for approx. 2 second.
2. After approx. 2 seconds, the device will enter into Forehead mode ➑ and it is ready for measurement - this will be indicated by a short beep.
3. To begin the measurement, point the sensor ➊ at the middle of the patient's forehead at a distance of approx. less than 50 mm. If the patient's forehead has hair, sweat or dirt on it, remove this first so that the reading is as accurate as possible.
4. Press the START button ➕ and release. After approx. 1 seconds, a short beep can be heard to indicate that the measurement is finished and the measured temperature will be shown. If the temple area has not been reached until the beep can be heard, repeat the measurement.

If the result is below 37.3°C, normal temperature indicator ➍ will be shown and the display will be lighted in Green for 3 seconds.

If the result is equal to or higher than 37.3°C and below 37.8°C., normal temperature indicator ➍ will be shown and the display will be lighted in Yellow for 3 seconds.

If the result is equal to or higher than 37.8°C, fever indicator indicator ➎ will be shown and the display will be lighted in Red for 3 seconds.

The device will automatically switch off if there is no input for approx. 60 seconds. You may also switch the device off manually by pressing the Power button ➋.

Measuring an object temperature

1. Hold the Mode button ➑ to switch between Forehead mode ➑ and Object mode ➒.
2. Aim the thermometer at the centre of the object and keep a distance of less than 50 mm. Press START button ➕ for approx. 1 second. The unit measures the temperature, emits an acoustic signal and displays the measured value.
3. Without further input, the unit switches off after approx. 60 seconds. You may also switch off the device manually by pressing the Power button ➋.

Change the measurement unit

You can measure the temperature in °C or in °F. Please proceed as follows to select the unit of measurement:

1. The device has to be switched off.
2. Long press the START button ➕ until you see the symbol «°C» or «°F» displayed on with a short beep.
3. Press the START button ➕ to change the unit of measurement.
4. When the preferred unit shown on the display, press the Power button ➋ to save the selected unit and exit the unit changing mode.

Recalling stored data

The thermometer can store up to 30 results. When the device has been turned on and ready for measurement, you can call up your stored data by pressing the Power button ➋ for approx. 3 seconds. The display will show «M» memory display symbol ➏. If measurement values are stored in memory, first the memory slot number «1» appears with the stored temperature. By further pressing the Power button ➋, you can call up more stored values (memory slot «2» etc.). Totally there are 30 memory slots available, which may be recalled by pressing the Power button ➋ repeatedly.

Error messages

Screen display	Reasons	Solutions
Er 1	The thermometer is not functioning properly.	Take out the battery. Wait for 1 minutes and insert it again. If the message reappears, please contact your dealer or service centre.
Er 2	The ambient temperature is not within the range of between 5°C and 40°C (41 F - 104 F).	Place the thermometer in a room for at least 30 minutes at room temperature between 5°C and 40 °C (41°F - 104°F).
Hi	Forehead mode: Measurement result is higher than 43°C (109.4°F). Object mode: Measurement result is higher than 100°C (212°F).	Read the measuring instructions thoroughly, then take a new temperature measurement.
Lo	Forehead mode: Measurement result is lower than 34°C (109.4°F). Object mode: Measurement result is higher than 0°C (32°F).	Read the measuring instructions thoroughly, make sure the sensor is clean and take a new temperature measurement.
	The thermometer works properly.	Use the thermometer normally.
	When the battery symbol flash, it indicates that the power is low but you can still measure.	The thermometer will work properly but batteries must be replaced soon.
	The thermometer could not work due to low power.	Replace with two new AAA alkaline batteries.

Cleaning and maintenance

- The sensor ➊ is the most delicate part of the device. Protect it from dirt and damage. Clean the sensor with a cotton bud dipped in warm water or 70% isopropyl alcohol.
- Do not immerse the thermometer in water or any other liquid. **It is not watertight.**
- To clean the thermometer, use a soft cloth slightly dampened with mild soapy water or 70% isopropyl alcohol. Do not let water get into the unit. Only use the unit again when completely dry.
- Never use aggressive cleaning agents, thinners, benzine or tough brushes.
- Take care not to scratch the lens of the sensor, the display or the housing.
- Do not use or store the unit in an environment where the ambient temperature or humidity is too high or too low (see operating conditions in the "Technical specifications" section), in direct sunshine, in the vicinity of static electricity or in a dusty environment.
- Remove the batteries if you do not wish to use the unit for an extended period of time. Otherwise there is a danger that the batteries may leak.
- If the thermometer is used as described in these instructions, no regular maintenance or calibration is required.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your supplier for information about disposal.

Technical specifications

Name and model:	medisana Clinical Infrared Thermometer DET-306
Display system:	digital display (LCD)
Power supply:	2 x batteries (AAA type, LR03) 1.5V
Expected service life:	3 years or 6,000 measurements
Measuring range:	Forehead 34-43°C (93.2-109.4°F)
Accuracy of measurement (laboratory readings):	±0.2°C, 35.5°C-42.0°C (±0.4°F, 95.9°F-107.6°F) , other range ±0.3°C (±0.5°F)
Resolution:	0.1°C (0.1°F)
Memory capacity:	up to 30 readings
Operating temperature:	5-40°C (59-104°F); rel. humidity 15-85%
Storage/transport temperature:	-25°C to +55°C (-13°F to +131°F), rel. humidity 15-95%
Dimensions:	approx. 153 x 41 x 44 mm
Weight:	approx. 84 g
Article No.:	48892
EAN Code:	40155 88 48892 4

The CE-mark of this device refers to the EC-directive 93/42/EEC MDD. The unit is designed for home use only. For commercial use, a metrological inspection must be carried out once every year. This calibration will be charged for and can be carried out by an appropriate authority or an authorised service centre - in accordance with the regulations for users of medical equipment.

Electromagnetic compatibility: The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. Details on this measurement data can be found at the end of this manual.

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for medisana products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

JOYTECH HEALTHCARE CO., LTD.
No.365, Wuzhou Road, 311100 Hangzhou, Zhejiang Province, PEOPLE'S REPUBLIC OF CHINA

Authorised EU Representative
Shanghai International Holding Corp. GmbH (Europe)
Address: Eiffestrasse 80, 20537 Hamburg, German

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