medisana®

RG 219

MMG Pro Fascia Power Gun

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# medisana®

Thank you for purchasing the MMG Pro and welcome to medisana!  
Please read this user manual carefully and keep it in a safe place for  
future reference.

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Product features	P.6
Safety instruction	P.7
Useful information	P.8
Battery charging	P.9
Operating instruction	P.9
Maintenance and caring	P.10
Disposal	P.10
Warranty and support	P.10
Specifications	P.11

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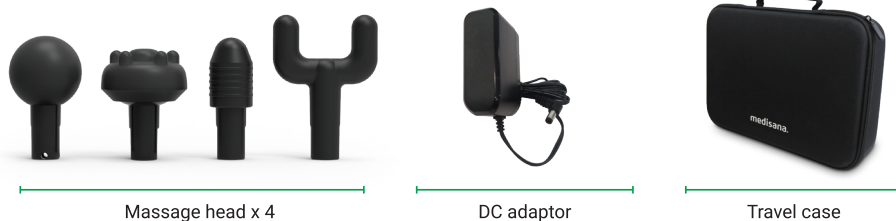
## PRODUCT FEATURES

### The set including:

- RG 219 Fascia Power Gun x 1
- Round-shape massage head x 1
- Bullet-shape massage head x 1
- Fork-shape massage head x 1
- Pebble-shape massage head x 1
- DC adaptor x 1
- Travel case x 1
- User manual x 1

## SAFETY INSTRUCTION

### IMPORTANT INFORMATION!

#### KEEP FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

#### INTRODUCTION

This instruction belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly.

#### WARNING

These warning notes must be observed to prevent any injury to the user.

#### CAUTION

These notes must be observed to prevent any damages to the device.

#### NOTE

These notes give you useful additional information on the installation or operation.

Connect the device for charging only in that way, so that the power plug is freely accessible.

Always switch off the appliance after use with the function button.

To disconnect the unit from the power supply, always pull the plug out of the socket. Do not pull on the cable!

Do not carry, pull or twist the device on the cable or the power supply unit.

You must not continue to use the unit if the lead or the mains adaptor are damaged. For safety reasons, these parts may only be replaced by authorized service personnel. In order to prevent hazards, always send the unit in to the service center.

Ensure that the cable will not cause tripping. The cable must not be kinked, trapped or twisted.

This appliance can be used by adult only and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.

Do not use this unit to supplement or replace medical treatment.

Chronic complaints and symptoms may be made worse.

Do not use the appliance near organs or near any other sensitive areas.

If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.

Before using the device, consult your doctor regarding any unexplained pain. If you are undergoing medical treatment or using medical equipment.

### Before Using the Unit

The device is not to be used for commercial use or medical purposes.

If you have health concerns, consult your doctor before using the device.

Only use indoors! Do not use the device in damp rooms (e.g. when bathing or showering)

The device must only be used for its intended purpose as described in the instruction manual. Using the unit for any other purpose invalidates warranty.

Do not use the appliance while you are asleep or when you are lying in the bed.

Do not use the appliance before going to sleep. The massage has a stimulating effect.

Never use the appliance while driving a vehicle or operating a machine.

The maximum operating time for one session with vibration is 10 minutes.

After this time is exceeded, the product automatically switches off.

Let the motor cool-down for several minutes, before you switch on the product again.

Never leave the unit unattended if it is connected to the power supply (for charging the batteries).

Do not allow the appliance to come into contact with pointed or sharp objects.

Do not cover up the appliance when it is switched on. Never use it under blankets or cushions. There is a risk of fire, electric shock or injury.

The integrated batteries are not designated for exchange or removal by the end user. If problems with the power supply should occur, contact the service center.

You may only clean the device yourself. If the device malfunctions, please do not repair it yourself as this leads to the automatic expiry of all guarantee rights. Ask your specialist dealer and have repairs carried out by the service. Cleaning and user maintenance shall not be made by children without supervision.

Do not immerse the device in water or other fluids.

If, nonetheless, liquid should penetrate the device, switch it off immediately and unplug the mains cable if it was connected. Wait until the appliance has dried completely.

## USEFUL INFORMATION

### Items Supplied and Packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service center.

The packing is being reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**⚠ WARNING**

Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

**Fascia Power Gun** is a massage device with vibration motor for the deeply efficacious vibration massage. It supports to prevent muscle tensions and to regenerate your muscles more quickly (e.g. after intense workout). **Fascia Power Gun** combines the deep vibration massage with a pressure massage (by your own weight), so that the tensed muscles will be effectively loosened.

**⚠ CAUTION**

Make sure that the maximum operating time of 10 minutes.

## BATTERY CHARGING

Before first use, the lithium battery should be charged (1 x lithium battery 7.4V, 2600 mAh) completely. To charge the battery fully, charging time should be approx. 3-4 hours.

Put the supplied 9V DC adaptor to the hole at bottom of **Fascia Power Gun**.

During charging, the LED light above Power On/Off button will flash red. After the battery have been fully charged, the LED light will glow constantly in blue. (Device cannot be used during charging)

**Fascia Power Gun** can work for around 1.5 hours after it's fully-charged.

**⚠ WARNING**

If the battery indicator hasn't flash after six hours continuously charging, plug out mains adaptor to avoid danger.

## OPERATING INSTRUCTION

Use only on a dry surface of the body over clothing by lightly pressing and moving across the body for approximately 60 seconds per region.

Use **Fascia Power Gun** only on the body's soft tissue as desired without producing pain or discomfort. Do not sure on the head or any hard or bony area of the body.

Use only the applicators that provide the best-desired effect.

Bruising can occur regardless of control setting or pressure applied. Check treated areas frequently and immediately stop at first sign of pain or discomfort.

Keep fingers, hair, or any other body part away from the shaft and backside of the applicator as pinching may occur.

Do not place any objects into the ventilation holes of the **Fascia Power Gun**.

Do not immerse in water or allow water to enter the ventilation holes of the **Fascia Power Gun**.

Do not drop or misuse the **Fascia Power Gun**.

Only recharge with supplied 9V DC adaptor.

Never leave the **Fascia Power Gun** operating or charging unattended.

**⚠ WARNING**

Only use the power adapter provided to power the product.

**⚠ WARNING**

This product is not suitable for use by people (including children) with reduced physical, sensory or mental capabilities, or a lack of experience and knowledge, unless they are supervised or have been instructed how to use the product safely by a person responsible for their safety.

**⚠ WARNING**

Children should be supervised to ensure that they do not play with the device.

## OPERATING INSTRUCTION

Select the appropriate detachable massage head for your need and screw into the **Fascia Power Gun**.

Power on by pressing the Power On/Off button for 3 second, then the LED at rear panel will light up in blue.

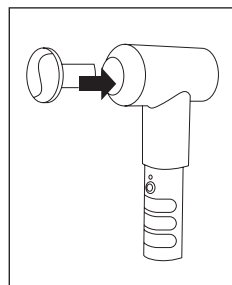
Vibration starts at LOW (2,800 rpm) level. You can press the button again to MEDIUM (3,400 rpm) level or switch to HIGH (3,900 rpm) level by press the button again.

Long press Power On/Off button 3 second to turn off the **Fascia Power Gun**.

Massage desired body part at the preferable speed pressure as needed.

**Fascia Power Gun** presets with 10-minute automatic power off timer.

Charge the device again when the vibration speed slows down.

## MAINTENANCE AND CARING

Before cleaning the appliance, make sure that the appliance is switched off and the power supply unit is disconnected. Allow the unit to cool down.

Clean the device with a slightly damp towel. Never use brushes, aggressive cleaning agents, petrol, thinners or alcohol. Rub the unit dry with a clean dry cloth.

Never immerse the appliance in water to clean it and ensure water cannot get into the appliance.

Wait until the device is complete dry before using it again.

Undo any kinks in the cable.

Take care when rolling up the mains lead to avoid damaging it.

You should ideally store the device in its original packaging and keep it in a clean, dry place.

Don't use the product for over 30 minutes continuously. It may activate the over-heat protection and stop operation automatically. Please wait at least 30 minutes in order to cool down before re-use.

## DISPOSAL

The battery in the appliance contains environmentally hazardous substances. Before the appliance is discarded, the power should be cut off and the battery removed, and then the battery should be discarded in a safe situation.

This product must not be disposed of together with domestic waste all users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

## WARRANTY AND SUPPORT

Please contact your local dealer or distributor in case of a claim under the warranty.

The following warranty terms apply:

1. The warranty period for **Fascia Power Gun** is one year from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a) Any losses or damages caused by improper treatment.
  - b) Any losses or damages caused by acts of God, civil commotion, wars, any accidents or any other beyond medisana's reasonable control.
  - c) All damage which is due to repairs or tampering by the customer or unauthorized third parties.
  - d) Damages caused by external connection with other equipment.
  - e) Accessories which are subject to normal wear.
5. If there is any specific service terms and conditions for a particular Repair Service provided by medisana, such specific service terms and conditions shall prevail in case of any discrepancy with these Terms of Repair Services.

If you need further information or support, please contact the local dealer or distributor in your country.

## SPECIFICATIONS

Model no.	RG 219
Product name	MMG Pro Fascia Power Gun
Article no.	48222
Battery type	Built-in rechargeable lithium
Power supply	Input: 100-240V, 50-60Hz Output: DC 9V 1A
Battery specification	7.4V, 2600 mAh
Operation time	Approx. 1.5 hours at HIGH speed level
Charging time	Approx. 3.5 hours
Automatic power-off	After approx. 10 minutes of use
Vibration speed	LOW 2800 rpm, MEDIUM 3400 rpm, HIGH 3900 rpm
Operation condition	Dry, indoor locations only
Storage condition	Clean, cool and dry location
Dimension	Approx. 23.5 x 6 x 19 cm (including massage head)
Weight	Fascia Power Gun: approx. 692 g (not including massage head) Massage head: approx. 19-40 g (depends on different head)

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## Medisana Far East Ltd.