
# Portable Fascia Power Gun with Hot & Cold Massage RG 220

## Instruction Manual

Please read carefully!

## GB Device and Controls

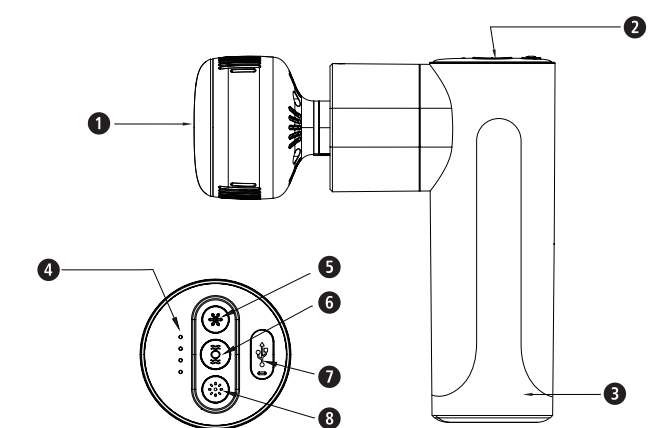### Explanation of symbols

**IMPORTANT**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damages to the device.

**NOTE**  
These notes give you useful additional information on the installation or operation.

LOT number

Serial number

Manufacturer

Date of manufacture

## GB Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use. This instruction belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly.

These notes give you useful additional information on the installation or operation.

- Connect the device for charging only in that way, so that the power plug is freely accessible.
- Always switch off the appliance after use with the function button.
- To disconnect the unit from the power supply, always pull the plug out of the socket. Do not pull on the cable!
- Do not carry, pull or twist the device on the cable or the power supply unit.
- You must not continue to use the unit if the lead or the mains adaptor are damaged. For safety reasons, these parts may only be replaced by authorized service personnel. In order to prevent hazards, always send the unit in to the service center.
- Ensure that the cable will not cause tripping. The cable must not be kinked, trapped or twisted.
- This appliance can be used by adult only and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Do not use this unit to supplement or replace medical treatment.
- Chronic complaints and symptoms may be made worse.
- Do not use the appliance near organs or near any other sensitive areas.
- If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.
- Before using the device, consult your doctor regarding any unexplained pain. If you are undergoing medical treatment or using medical equipment.

### Before Using the Unit

- ⚠ The device is not to be used for commercial use or medical purposes.
- 👨‍⚕️ If you have health concerns, consult your doctor before using the device.
- 🏠 Only use indoors! Do not use the device in damp rooms (e.g. when bathing or showering)

- The device must only be used for its intended purpose as described in the instruction manual.
- Using the unit for any other purpose invalidates warranty.
- Do not use the appliance while you are asleep or when you are lying in the bed.
- Do not use the appliance before going to sleep. The massage has a stimulating effect.
- Never use the appliance while driving a vehicle or operating a machine.
- The maximum operating time for one session with vibration is 15 minutes.
- After this time is exceeded, the product automatically switches off.
- Let the motor cool-down for several minutes, before you switch on the product again.
- Never leave the unit unattended if it is connected to the power supply (for charging the batteries).
- Do not allow the appliance to come into contact with pointed or sharp objects.
- Do not cover up the appliance when it is switched on. Never use it under blankets or cushions. There is a risk of fire, electric shock or injury.
- The integrated batteries are not designated for exchange or removal by the end user. If problems with the power supply should occur, contact the service center.
- You may only clean the device yourself. If the device malfunctions, please do not repair it yourself as this leads to the automatic expiry of all guarantee rights. Ask your specialist dealer and have repairs carried out by the service.
- Cleaning and user maintenance shall not be made by children without supervision.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, switch it off immediately and unplug the mains cable if it was connected. Wait until the appliance has dried completely.

## Unit and Display

### Main unit

- 1 Message head
- 2 Control panel
- 3 Battery compartment

### Control panel

- 4 LED battery indicator
- 5 Cold massage on/off button
- 6 Vibration on/off button
- 7 Charging port
- 8 Hot massage on/off button

### Massage head

- 9 Hot & cold massage head
- 10 Cylindrical massage head
- 11 Ball massage head
- 12 Shovel massage head
- 13 Bow massage head
- 14 Flat massage head
- 15 U-shaped massage head

## Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service center. The packing is being reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

The following parts are included:

- 1 **medisana** Portable Fascia Power Gun with Hot & Cold Massage **RG 220**
- 1 Hot & cold massage head
- 1 Cylindrical massage head
- 1 Bow massage head
- 1 U-shaped massage head
- 1 Portable storage case
- 1 Ball massage head
- 1 Shovel massage head
- 1 Flat massage head
- 1 USB to Type-C charge cable
- 1 instruction manual

Please contact your supplier if you notice any transport damage on unpacking the unit.

### WARNING

Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

Portable Fascia Power Gun with Hot & Cold Massage is a device for deeply relaxing muscular fascia and suitable for a physical massage after exercise and fitness. It can alleviate stiffness and ache of muscles, relieve muscle fatigue, and accelerate blood circulation to improve the soft tissue of the body.

### CAUTION

Make sure that the maximum operating time of 15 minutes.

## Operating instruction

- Use only on a dry surface of the body over clothing by lightly pressing and moving across the body for approximately 60 seconds per region.
- Use only on the body's soft tissue as desired without producing pain or discomfort. Do not sure on the head or any hard or bony area of the body.
- Use only the applicators that provide the best-desired effect.
- Bruising can occur regardless of control setting or pressure applied. Check treated areas frequently and immediately stop at first sign of pain or discomfort.
- Keep fingers, hair, or any other body part away from the shaft and backside of the applicator as pinching may occur.
- Do not place any objects into the ventilation holes of the device.
- Do not immerse in water or allow water to enter the ventilation holes of the device.
- Do not drop or misuse the device.
- Only recharge with input voltage: 5V, 2A.
- Never leave the device operating or charging unattended.

### WARNING


This product is not suitable for use by people (including children) with reduced physical, sensory or mental capabilities, or a lack of experience and knowledge, unless they are supervised or have been instructed how to use the product safely by a person responsible for their safety.

### WARNING

Children should be supervised to ensure that they do not play with the device.

## Massage head installation


1. Disassembly and assembly of the massage head are conducted in a plug-in way, free from screwing;
2. Fix the red vertical bar of the hot & cold massage head on the piston notch;
3. Make sure it is inserted to the bottom tightly.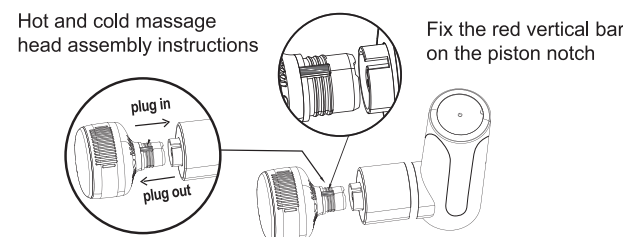


## Charging the device

The charging port 7 is located on the top panel of the handle.

1. Charge the battery before using the product for the first time. To charge the battery fully, charging time should be approx. 5 hours. The accessory TYPE-C USB cable must be used for recharging the device's battery. The other end is equipped with any 5 volts charger adapter (5V and 2A is recommended).
2. When charging begins, the LED battery indicator 4 on device flash.
3. After the battery have been fully charged, the LED battery indicator 4 will glow constantly.
4. Battery performance is influenced by the surrounding temperature. Charge the supplied rechargeable batteries within a temperature range of 0-40°C.
5. Charge timely when the battery is lower than 25%.
6. It is not recommended to completely run out of the battery to 25%.
7. Average running time is 3 hours after the device is charged, which depends on vibration level, hot or cold massage.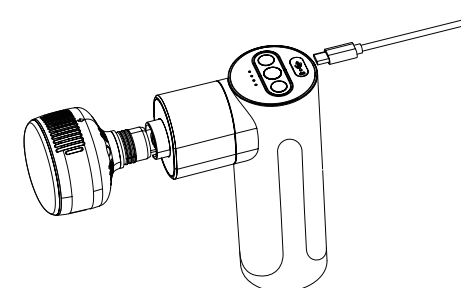

## Operation

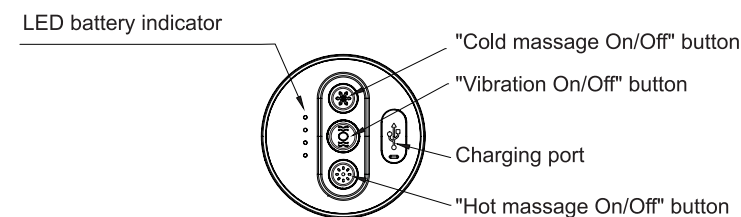1. Press any buttons when the device is power-off, the LED battery indicator 4 shows the current battery level.
2. Vibration mode: long press the "Vibration on/off" button 6 for 3 seconds, the vibration function will start. When the LED light turns to Blue, Level 1 vibration is started. Short press the button "Vibration on/off" button 6, LED light turns to purple and Level 2 vibration starts. Short press the "Vibration on/off" button 6 to switch to Level 3 vibration, LED light turns to red. You can long press the "Vibration on/off" button 6 to turn off the vibration function at anytime.
3. Hot massage mode: long press the "Hot massage On/Off" button 8 for 3 seconds, the hot massage function will start at about 35°C and the LED light turns to Blue. Short press the button "Hot massage On/Off" button 8, LED light turns to purple and increase to 40°C. Short press the "Hot massage On/Off" button 8, LED light turns to red and increase to 45°C. You can long press the "Hot massage On/Off" button 8 for 3 seconds to turn off the hot massage function at anytime.
4. Cold massage mode: long press the "Cold massage On/Off" button 5 for 3 seconds, the cold massage function will start at about 10°C and the LED light turns to Blue. Short press the button "Cold massage On/Off" button 5, LED light turns to purple and increase to 15°C. Short press the "Cold massage On/Off" button 5, LED light turns to red and increase to 20°C. You can long press the "Cold massage On/Off" button 5 for 3 seconds to turn off the cold massage function at anytime.
5. When any modes is activated, you can short-press the buttons to start corresponding function.
6. When all the functions turned off, the LED battery indicator 4 will glow for 7 seconds constantly and the device is at standby mode. The device will automatically power off after 5 seconds if no further instructions.
7. Only hot & cold massage head is applicable for hot or cold massage mode and it must be installed tightly before use.
8. Vibration, hot massage and cold massage can be operated independently.
9. The device switches off automatically after 15 minutes of use.

## Maintenance and caring

- Before cleaning the appliance, make sure that the appliance is switched off and the power supply unit is disconnected. Allow the unit to cool down.
- Clean the device with a slightly damp towel. Never use brushes, aggressive cleaning agents, petrol, thinners or alcohol. Rub the unit dry with a clean dry cloth.
- Never immerse the appliance in water to clean it and ensure water cannot get into the appliance.
- Wait until the device is complete dry before using it again.
- Undo any kinks in the cable.
- Take care when rolling up the mains lead to avoid damaging it.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.
- Don't use the product for over 30 minutes continuously. It may activate the over-heat protection and stop operation automatically. Please wait at least 30 minutes in order to cool down before re-use.

## Disposal

The battery in the appliance contains environmentally hazardous substances. Before the appliance is discarded, the power should be cut off and the battery removed, and then the battery should be discarded in a safe situation.

This product must not be disposed of together with domestic waste all users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

## Specifications

Product name and model no.	: <b>medisana</b> Portable Fascia Power Gun with Hot & Cold Massage <b>RG 220</b>
Article no.	: 48939 (Grey), 48940 (Blue), 48858 (Black)
Battery type	: Built-in rechargeable lithium
Input voltage	: 5V, 2A
Battery specification	: 11.1V, 2400 mAh
Operation time	: Cold massage (Level 3): approx. 2.5-3 hours : Hot massage (Level 3): approx. 8-8.5 hours : Vibration massage (Level 3): approx. 3-3.5 hours : Vibration with cold massage (Level 3): approx. 1-1.5 hours : Vibration with hot massage (Level 3): approx. 2-2.5 hours : Approx. 4.5-5 hours for fully-charge
Charging time	: After approx. 15 minutes of use
Automatic power-off	: Level 1: 2,800 rpm; Level 2: 3,000 rpm; Level 3: 3,200 rpm
Vibration speed	: For battery capacity (20%, 50%, 100%) and charging
LED battery indicator	: Dry, indoor locations only
Operation condition	: Clean, cool and dry location
Storage condition	: Approx. 14 x 6 x 15 cm (including massage head)
Dimension	: Body: approx. 620 g
Weight	: Hot & Cold massage head: approx. 124 g, 6 different massage head: 14-20 g

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

## Warranty And Support

Please contact your local dealer or distributor in case of a claim under the warranty.

The following warranty terms apply:

1. The warranty period for the device is one year from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a) Any losses or damages caused by improper treatment.
  - b) Any losses or damages caused by acts of God, civil commotion, wars, any accidents or any other beyond Medisana's reasonable control.
  - c) All damage which is due to repairs or tampering by the customer or unauthorized third parties.
  - d) Damages caused by external connection with other equipment.
  - e) Accessories which are subject to normal wear.
5. If there is any specific service terms and conditions for a particular Repair Service provided by Medisana, such specific service terms and conditions shall prevail in case of any discrepancy with these Terms of Repair Services.

If you need further information or support, please contact the local dealer or distributor in your country.

**medisana.**

Unit E&F, 31/F, YHC Tower, 1 Sheung Yuet Road, Kowloon Bay, Kowloon, Hong Kong

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