

# medisana®

GB Instruction manual  
Fascia Power Gun RG 221



#### Explanation of symbols



#### IMPORTANT

Failure to follow these instructions may result in serious injury or damage to the device.



#### WARNING

These instructions must be followed to prevent possible injury to the user.



#### CAUTION

These instructions must be followed to prevent possible damage to the device.



#### NOTE

These instructions provide useful additional information regarding installation and operation.



LOT number



Serial number



Manufacturer



Date of manufacture



Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.



#### WARNING!

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

## GB Safety Instructions



Please read these instructions carefully before using the device, in particular the safety instructions. Keep the instructions in a safe place for future reference. If you pass the device on to third parties, be sure to include these instructions for use.



- The device is intended for private use only. The appliance must not be used for commercial or medical purposes.
- If you have any health concerns, consult your doctor before use.
- Use the unit indoors only! Do not use the device in damp rooms (e.g. when bathing or showering).
- Use the device only for its intended purpose according to the instructions for use. In case of misuse, the warranty becomes void.

#### For power supply

- Before connecting the unit to the power supply, make sure that it is switched off and that the electrical voltage indicated on the label matches your mains voltage.
- Keep the charging adapter, cable and device away from heat, hot surfaces, moisture and liquids.
- Do not touch the charging adapter when standing in water or when your hands are wet or damp.
- Only charge the unit's battery with the supplied charging adapter EH1260-0500WA (12.6V, 0.5A).
- The unit must be connected so that the mains plug is freely accessible.
- To disconnect the unit from the mains, never pull on the mains cable but always on the mains plug.
- Do not carry, pull or turn the appliance by the mains cable.
- If the mains cable is damaged, it must only be replaced by the manufacturer, an authorised dealer or a suitably qualified person.
- The unit's cable can become a hazard (strangulation, tripping) if it is not routed correctly. The user must ensure that the power cable is laid safely.
- If you feel pain or find the massage uncomfortable, stop the application and consult your doctor.
- You must not use the device if you suffer from one or more of the following illnesses or complaints: Circulatory disorders, varicose veins, open wounds, bruises, skin tears, phlebitis or thrombosis.
- Do not use the appliance if you are pregnant. Special care is necessary when the appliance is used on, by or near children, sick and helpless persons.
- Do not use the device near the eyes or other sensitive parts of the body.
- Do not use this device to support or replace medical applications. Chronic conditions and symptoms could worsen.
- Do not use the device while you are sleeping or in bed.
- Do not use the device before falling asleep. The massage has a stimulating effect.
- Do not use the device while driving a vehicle or operating machinery.
- This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product in a safe way by a person responsible for their safety.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children must not play with the appliance.
- Cleaning and user maintenance must not be carried out by children without supervision.
- Do not allow the unit to come into contact with pointed or sharp objects.
- Do not cover the unit when it is switched on. Never use it under blankets or pillows. There is a risk of fire, electric shock and injury.
- Do not immerse the unit in water or other liquids.
- If liquid does enter the unit, switch it off immediately and disconnect the power supply if it was connected. Wait until the unit is completely dry.
- Only connect the unit for charging so that the cable and power connector are freely accessible.
- Never leave the unit unattended when it is connected to the mains (for charging) or in operation.

- The built-in batteries are not intended for replacement or removal by the end user. If you experience problems with the power supply, contact customer service.
- Always switch off the appliance after use by switching "Power On/Off" ① to "Off".
- To disconnect the appliance from the mains, always pull the plug out of the power socket. Do not pull on the cable!
- Do not carry, pull or twist the unit by the cable or the power plug.
- Make sure that the cable does not pose a trip hazard. Do not bend, pinch or twist the cable.
- Do not use the appliance if any damage is visible, if it is not working properly, if it has been dropped or immersed in water. To avoid hazards, send the unit to the service centre for repair.
- Do not insert any objects into the openings of the unit.
- You may only clean the unit. In case of malfunctions, please do not repair the unit yourself, as this will automatically void all warranty claims. Contact your specialist dealer and have the repairs carried out by the authorised service.
- Only use accessories intended for the unit and supplied by the manufacturer.
- Excessive use can lead to overheating. After 60 minutes of use, allow the appliance to cool down for **at least 30 minutes** before using it again.



#### ATTENTION

Make sure that the maximum operating time of 60 minutes is not exceeded!

#### Scope of delivery

Please first check whether the scope of delivery is complete and that the item is not damaged in any way. If in doubt, do not operate the unit and contact your dealer or service centre.

The scope of delivery includes:

- 1 **medisana** Massage Gun with Hot & Cold function **RG 221**
- 6 Massage heads
- 1 AC/DC adapter
- 1 Instruction manual
- 1 Storage case



Packaging is recyclable or can be returned to the raw material cycle. Please dispose of packaging material that is no longer required properly. If you notice any transport damage when unpacking, please contact your dealer immediately.

#### Device and controls

- ① Power On/Off switch
- ② Charging port
- ③ Intensity indicator (1-30 levels)
- ④ Battery indicator
- ⑤ "+" button for accelerate intensity
- ⑥ "-" button for decelerate intensity
- ⑦ 60 mm ball massage head
- ⑧ 50 mm ball massage head
- ⑨ Spade-shaped massage head
- ⑩ U-shaped massage head
- ⑪ Conical massage head
- ⑫ Flat massage head
- ⑬ AC/DC adapter

#### Charging the battery

- Before first use, the lithium battery should be charged (1 x lithium battery 11.1V, 1500mAh) completely. To charge the battery fully, charging time should be approx. 2 hours.
- Make sure the device is power-off when charging.
- Put the supplied AC/DC adapter to the hole at bottom of Fascia Power Gun.
- During charging, the LED light on and the AC/DC adapter will show red light. After the battery have been fully charged, the LED light on adapter will show green color. (Device cannot be used during charging)
- Fascia Power Gun can work for around 1.5 hours after it's fully-charged.

#### Basic operation

1. Push the "Power ON/OFF switch" ① left to power on the device, or push right to turn off the device.
2. Press the "+" button ⑤ to accelerate intensity level (01-30). Press the "-" button ⑥ to decelerate intensity level.
3. Battery indicator ④ shows the remaining battery life.

#### What are the other massage heads recommended for?

- 60 mm ball massage head ⑦: suitable for large muscle groups
- 50 mm ball massage head ⑧: suitable for large muscle groups
- Spade-shaped massage head ⑨: suitable especially for arm and leg muscles
- U-shaped massage head ⑩: suitable for muscles in the area of the spine, achilles tendon, ankle and calf

- Conical massage head ⑪: suitable for use in specific areas (trigger points, small muscle groups)
- Flat massage head ⑫: suitable for relaxation massage or loosening of large muscles

#### Cleaning and care

- Before cleaning the item, disconnect the power supply if necessary and allow the appliance to cool for at least 15 minutes.
- Clean the unit with a soft cloth slightly moistened with water. Never use harsh cleaning agents, petrol, thinner or alcohol.
- Then dry the unit with a dry, clean cloth.
- Do not expose the unit to direct sunlight, protect it from dirt and moisture.
- Make sure the unit is completely dry before using it again.
- Store the unit in a clean and dry place.

#### Disposal



This appliance must not be disposed of with household waste. Every consumer is obliged to hand in all electrical or electronic devices, regardless of whether they contain pollutants or not, to a collection point in his or her city or to a retailer so that they can be disposed of in an environmentally friendly manner. Remove the battery before disposing of the device. Do not dispose of used batteries/battery packs in household waste, but in hazardous waste or at a battery collection point in a specialist shop. Contact your local authority or dealer regarding disposal.

#### Technical specifications

Name and model:	<b>medisana</b> Fascia Power Gun <b>RG 221</b>
Battery:	Li-ion battery 11.1V, 1,500mAh
Power supply:	Input: 100-240V 50/60Hz Output: DC12.6V 0.5A
Operation time:	Approx. 1.5 hours
Charging time:	Approx. 2 hours
Intensity levels:	1,200-3,200 rpm (30 levels)
Operating/charging conditions:	Use / charge indoors only
Storage conditions:	Clean, cool and dry
Dimensions:	approx. 23 x 17 x 6.3cm
Weight:	approx. 750 g (incl. cable and massage heads)
Article number:	48238
EAN number:	40155 88 48238 0

In the interest of continued product improvement, we reserve the right to make technical and design changes.

#### Warranty and repair conditions

Your statutory warranty rights are not restricted by our guarantee below. In case of warranty please contact your specialist shop or the service centre directly. Should you have to return the device, please state the defect and enclose a copy of the purchase receipt or invoice.

The following warranty conditions apply:

1. All **medisana** products are guaranteed for 3 years from the date of purchase. In case of a warranty claim, the purchase receipt or invoice must be provided as proof of purchase date.
2. Faults due to material or manufacturing defects will be rectified free of charge within the warranty period.
3. A warranty service does not extend the warranty period for either the equipment or for replacement components.
4. The following are excluded from the warranty:
  - a. any damage caused by improper use, e.g. not following the user instructions.
  - b. damage caused by repair or intervention by the purchaser or unauthorised third parties.
  - c. transport damage that has occurred on the way from the manufacturer to the consumer or has arisen during transit to the service point.
  - d. spare parts that are subject to normal wear and tear.
5. Liability for direct or indirect consequential damages caused by the device is also excluded if the damage to the device is recognised as a warranty claim.

# medisana®

Unit E&F, 31/F, YHC Tower, 1 Sheung Yuet Road, Kowloon Bay, Kowloon, Hong Kong