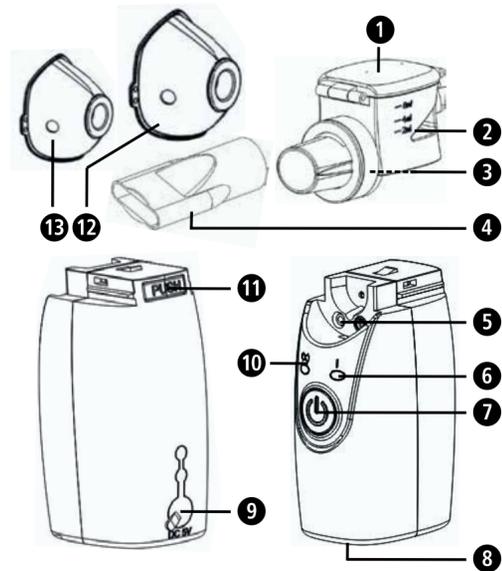
## Ultrasonic inhalator IN 525

Instruction Manual  
Please read carefully!

### GB Device and Controls

#### Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes give you useful additional information on the installation or operation.

Device classification: type BF applied part

LOT number

Manufacturer

Date of manufacture

Serial number

IP22 Protection Rating regarding foreign objects and drops of water

Protect against high temperatures!

### GB

#### IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

#### Intended use

- The **medisana** ultrasonic inhalator IN 525 is suitable for the temporary oral or nasal inhalation therapy with nebulised medication in private households or in a clinical environment. The unit must only be used by patients, who became familiar with the usage of the device resp. who were instructed or are supervised by medical professionals.
- The user must have read and understood these instructions of use and should have understood the general machine operation.
- The device may be used by persons, who suffer from asthma, chronic lung diseases (e.g. COPD, bronchitis or emphysema) or other respiratory diseases, which are characterized by obstructive air flow.

Use the device only according to its intended purpose as specified in the instruction manual. The warranty will be invalidated if used for purposes other than those for which it is intended.

- Any application other than the herewith described is improper and therefore considered dangerous. If the appliance is not used according to the instructions, the user is liable for the safe operation of the unit.
- Only use genuine accessories and spare parts provided by the manufacturer.
- Use only liquid medications that have been approved for use in inhalation therapy. Before starting therapy, discuss the length, dosage, frequency of use and the selection of the medication with your doctor or pharmacist.
- The nebulization intensity fluctuates depending on the used medication. Do not use only water in the nebulizer for nebulizing purposes.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children must be supervised to ensure that they do not play with the device.
- Cleaning and user maintenance shall not be made by children without supervision.
- Small parts can be swallowed. Therefore keep and dispose of all small parts out of reach of children.
- This device may become unusable due to voltage fluctuations, empty batteries or mechanical failure. We therefore recommend to always have spare batteries resp. a spare device available.
- If the device does not switch off automatically when the tank is empty, press the ON/OFF-button to switch it off manually.
- Disinfect all parts of the nebulizer including mask before first use, after each use and after a long period of non-usage (see point "Cleaning and maintenance").

**WARNING**  
Danger of infection from contaminated nebulizer unit!  
Follow the hygiene regulations (e.g. thoroughly washing the hands) before each use and make sure that the nebulizer unit has been cleaned and disinfected before first and after each use according to point "Cleaning and maintenance".  
Do not share one device with other persons to avoid the risk of infection!

- If the unit is visibly damaged or if you assume a defect, do not use the appliance. Contact the service centre.
- If you want to connect the device to your power supply with the optional adaptor, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Do not touch the adaptor or the adaptor cable with wet hands.
- Do not drop the device.
- Make sure, that all parts are correctly assembled before you use the appliance.
- Do not attempt to repair the device yourself in the event of a malfunction. Repairs should only be carried out by authorised service centres.
- Do not open or modify the device.
- Never expose the device to direct sunlight or extreme temperatures.
- Never try to clean the mesh with any objects - this could lead to damages!
- Never hold the device or parts of the device under running water for cleaning. Pay attention, that no liquid can reach the inner parts of the device.
- Never transport or store the device, if still liquid is inside of the tank. This could leak and damage the device.

#### Unit and LCD display

##### Nebulizer unit:

- Cover for active ingredient tank
- Mesh
- Active ingredient tank
- Mouthpiece

##### Device base:

- Contact electrodes
- Power indicator
- ON/OFF-button
- Battery compartment
- Jack for adaptor cable
- Battery indicator
- Release button for nebulizer unit
- Adult mask
- Child mask

#### Scope of supply

Please check first of all that the unit is complete.

The following parts are included as standard:

- 1 **medisana** Ultrasonic inhalator IN 525
- 1 Child and 1 adults' mask
- 2 Batteries (size AA/LR6, 1.5V)
- 1 Mains adaptor
- 1 Storage bag
- 1 Instruction manual

Please contact your supplier if you notice any transport damage on unpacking the unit.

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**  
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

#### Important information about inhalation therapy

Living = breathing / Breathing = living

Often, diseases of the respiratory system or the lung are lengthy, painful and life restricting. Timely, effective therapies such as inhalation therapy with aerosols can be of great help and assistance. A gas + particle mixture is defined as an aerosol; mist is an example of a natural aerosol. Aerosol therapy delivers medicines in an "air transported" form (e.g. nebulized) which are inhaled. The great advantage of this therapy (unlike taking tablets for example) is that the medicines are administered at the precise point where they will have greatest effect (via the oro-nasalpharyngeal cavity into the lung) without passing through other organs:

- The effect is noticeably rapid.
- There is no (significant) effect or side effect on other organs.
- Small quantities of medicines are used.

#### Ultrasound technology

An aerosol is generated in the ultrasonic nebulizer due to the vibrations of a piezoelectric crystal. Thus, electric energy is transformed into mechanical oscillations which create droplets. The optimum diameter of the droplets is 1 - 5 µm (according to «Guidelines for the aerosol therapy of lung diseases», Lungenliga Schweiz). The droplets can only penetrate into the narrow branches of the upper and lower airways if they have a small diameter thus transporting and depositing the medicine where it intended to be most effective.

#### Important information when using preparations / medications

##### NOTES

- Consult your doctor before using any medicinal products or medicines to ensure that they are suitable for your needs or alternatively only use the medicinal products/medicine prescribed for you by your doctor.
- The **medisana IN 525** ultrasonic inhaler is suitable for use with all common aerosol products (aerosols) with the exception of:
  - Products with solid constituents (suspensions)
  - Viscous solutions (high viscosity)
  - Volatile oils
- Do not use any volatile oils since this may lead to damage to the nebulizer mesh.
- Use only water-soluble products/medicines or those which have an alcohol base.
- Use only isotonic saline solution to dilute the inhaled; otherwise bronchial spasms may occur in certain circumstances.
- Do not use oil-based inhalants.
- After using the device for salt solution inhalation, it must be cleaned and rinsed out, according to section "Cleaning and maintenance".

#### Accessories

There is no provision for combining with other devices. Use only the original accessories with this device since only then can safe operation be guaranteed.

#### Leakage safety

When pouring the inhalant into the active ingredient tank , only fill it to the maximum mark (8 ml). The recommended filling quantity is between 0.5 and 8 ml. During inhalation, you may tilt the device for up to 15 seconds to the front, rear or lateral side, without impairing the nebulisation effect or the treatment success. If - however - the contact between medication and mesh is interrupted for more than 15 seconds, the device will automatically switch off.

The **medisana IN 525** is leakage proof. Do not, however, shake the device when the active ingredient tank is full; there is a chance that liquids could get into the electronics and destroy the device.

#### Operating

##### Inserting / removing batteries

Open the battery compartment on the underside of the unit by exerting slight pressure whilst sliding it simultaneously in the direction of the arrow. Insert the two supplied batteries (type AA LR6). Remember to insert the batteries as indicated in the illustration on the base of the device. Replace the lid, pressing it until it snaps into place. If the battery indicator lights up in orange when the device is switched on, replace the batteries. If the indicator flashes orange, the device may still be used for approx. 30 minutes.

##### **WARNING** BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge or short circuit these batteries! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

#### Using a mains adaptor

Alternatively, you can use the unit with a special mains adaptor (**medisana** Art.-No. 54118 (EU) or 54119 (GB)). This connects to the jack on the unit. In this case the batteries should be removed from the device. The appliance is not suitable to charge batteries. We recommend to follow the points mentioned in chapter "Preparation" before connecting the device to the mains adaptor.

#### Preparation

- Make sure, the device is switched off (power indicator is not illuminated).
- Separate the nebulizer unit from the device base by pressing the release button and sliding the nebulizer unit to the front.
- Open the cover of the active ingredient tank and fill the tank with inhalant (maximum capacity 8 ml = 8 ccm). Please observe the dosage and instructions for use on the information leaflet enclosed with the medicine you are using.
- Close the cover of the active ingredient tank .
- Place the nebulizer unit onto the device base appropriately and lock it (it clicks into place).

#### Inhalation using the mouthpiece

- Sit as upright as possible.
- Put the mouthpiece onto the device base and enclose the mouthpiece completely with your lips.
- Switch on the device by pressing the ON/OFF button . The power indicator lights up in green. The aerosol warms up slightly during operation and the tank will light up in blue.
- Now breathe in slowly and deeply through your mouth and out through your nose. You can improve the effectiveness of the therapy, if you hold your breath for a short time after breathing in. However, this is not necessary for general success of the therapy. It is only important that you keep quiet and relaxed while breathing. Do not inhale too fast. If you need a break, stop using the device for a short time and take the mouthpiece out of your mouth. Then put it into your mouth again (lips enclosing it tightly) and breathe slowly in and out again.

##### NOTES

- Hold the device as upright as possible. A slight tilt does not affect its use (leakage proof). However, take care during inhalation to only tilt the device slightly in any direction.
  - If you tilt the device too much, the medication may lose sufficient contact to the membrane. In this case, the device will automatically switch off after approx. 15 seconds.
  - Tilt the device slightly when the remaining aerosol is almost exhausted to make sure the residual quantity will be spread through the mesh and sprayed completely.
- An inhalation application should not exceed 30 minutes.
  - When the tank is empty, the power indicator lights up in orange and the device will switch itself off after a short period of time. You may also switch off the device by pressing the ON/OFF button after terminating the inhalation, even if you have not used up all the inhalant. The power indicator expires.
  - Pour out the rest of the inhalant. Do not use it again. Clean the device immediately after each use (also refer to "Cleaning and maintenance").

Once you have used the mouthpiece for a certain time, two or three drops of inhalant solution may gather in the mouthpiece pipe. However, this does not affect the operation and function of the device. Wipe off the drops with a soft cloth after use.

#### Inhalation using the mask

Alternatively, a child and an adult inhalation mask is supplied with the device. These masks may be used for inhaling aerosol through the nose. This method is particularly suitable for elderly people.

##### NOTES

A larger amount of aerosol remains on the mucosa of the nasal pharynx when using the mask than when using the mouthpiece.

For the inhalation with the mask, plug the adult mask or the child mask onto the outlet of the ingredient tank. Afterwards, follow the steps mentioned in chapter "Inhalation using the mouth-piece". The difference is, that the mouth does not need to be put on the mouthpiece, but instead needs to be pressed lightly over your mouth and nose. Make sure, that the mask seals around mouth and nose without being too tight. Now breathe in and out slowly and deeply through your nose.

#### Constant use

For a constant use of this device we recommend, to use it 3 times per day with 10 minutes for each session (with mask or mouthpiece) at a room temperature of approx. 23°C. Under these circumstances, the main unit provides 3 years and the nebulizer unit provides 6 months of a durable period of service. The nebulizer unit is a part of wear and tear and is not covered by the warranty. The nebulizer unit may be reordered under art. no. 54116.

#### Cleaning and maintenance

##### NOTES

Carry out the cleaning instructions carefully directly after each use as described below.

- Remove the mask if applicable.
- When cleaning, do not hold the whole device or the nebulizer unit under running water. Take care to ensure that no water gets inside the device.
- Clean the device's casing with a lightly moistened cloth that has been soaked in mild soapy water.
- Clean the mask under running water.
- Clean the electrodes of the device base and the nebulizer unit carefully with a smooth, slightly dampened cloth.
- The base and the nebulizer unit can be cleaned in the following way:
  - Pour out any residual inhalant in the active ingredient tank .
  - Fill some drinking water into the tank.
  - Turn on the device to nebulize the distilled water for 1 to 2 minutes.
  - Now take out the batteries (resp. disconnect the power supply) and remove the nebulizer unit from the device base.
  - Clean the nebulizer unit by hand with distilled water.
  - Afterwards let all parts completely dry on the air on a smooth, dry padding (e.g. a towel).
- This cleaning routine should also be carried out if the device has not been used for a longish period of time and prior to using for the first time.

##### **WARNING**

- Keep the battery compartment dry all the time!
- Never touch the mesh with a finger or objects!
- Do NOT clean the device or any parts in a dishwasher!
- Do NOT use microwave ovens to dry any parts!

#### Disinfection of the nebulizer unit

- In the case of infectious illnesses, you need to follow below steps to disinfect the nebulizer unit additionally. We recommend, that this measure is performed regularly either in the morning or the evening at a fixed time:
- Rinse the nebulizer unit with distilled water.
  - Boil some distilled water in a pot.
  - As soon as the water is boiling, take the pot away from the cooking area and put the nebulizer unit into the water for approx. 10 minutes.
  - Remove the nebulizer unit from the water and let it dry and cool down on the open air (e.g. on a towel).
  - Alternatively (instead of using distilled water) you may disinfect the nebulizer unit with a solution of 75% cleaning alcohol - with this, the time to put the unit into the liquid reduces to 1 minute only. Afterwards, rinse the unit with distilled water and let it dry on the open air.
  - Make sure that no disinfectant residues are left behind on the parts of the device to ensure safe inhalation when next used.

##### **WARNING**

- NEVER put the nebulizer unit directly into the boiling water when the pot is still on the cooking area!
- Solutions with alcohol are highly flammable. Therefore, NEVER clean the device near open fire or smoke.

#### Transportation and storage

- Store the appliance with all parts in a dry and clean place.
- Make sure, that no residual liquids are in the ingredient tank during storage or transportation.
- Remove the batteries if you do not use the device for a longer time.
- Avoid storage locations with direct sun exposure, extreme temperatures and high humidity.
- Always keep the device away from fire, electromagnetic radiation and from children.

#### Trouble-Shooting

In case of malfunctions please pay attention to the following points:

**Problem:** No or low nebulization

**Solution:** Replace the batteries, clean the electrodes with a slightly moistened, smooth cloth and clean the appliance according to chapter "Cleaning and maintenance". Check, whether the nebulizer unit is correctly mounted on the device base and if there is sufficient contact between inhalant and mesh (hold device straight).

**Problem:** No nebulization and power indicator does not light up or lights up only shortly.

**Solution:** Check, if the batteries have been inserted correctly and if these provide enough power. If in doubt, exchange the batteries. Check, whether the nebulizer unit is correctly mounted on the device base and if there is sufficient contact between inhalant and mesh (hold device straight).

**Problem:** The device switches itself off during, although there is still inhalant in the tank.

**Solution:** The mesh is defective or there is no correct connection between nebulizer unit and device base. Clean the electrodes with a slightly moistened, smooth cloth and check, if the nebulizer unit has been mounted onto the device base correctly and if there is sufficient contact between inhalant and mesh (hold device straight). Exchange the nebulizer unit.

**Problem:** Inhalant leaks off the device.

**Solution:** The silicone seal could be defective. Check, if the nebulizer unit is correctly mounted on the device resp. exchange the nebulizer unit.

If above points do not help to solve the problem, contact the customer service.

#### Directives / norms

This inhalation device meets the requirements of the EU standard 93/42/EWG (medical device directive) and of the standard EN 13544-1: 2007 Respiratory therapy equipment - Part1: Nebulising systems and their components.

#### Electromagnetic compatibility:

The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. You can find details on this measurement data on the separate enclosure.

#### Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

#### Technical specifications

Name and model:	<b>medisana</b> Ultrasonic inhalator IN 525
Power supply:	3V=, 2 x 1,5V batteries AA LR6 AC adaptor (model: TAP5-050S100U1; Input: 100-240V~, 50/60Hz, 0.15A; Output: 5V=, 1A)
Power consumption:	approx. 1.1W
Ultrasound frequency:	approx. 108 kHz
Nebulizing output:	≥ 0.2 ml/min.
Particle size:	MMAD approx. 5 µm
Filling volume inhalant:	min. 0,5 ml / max. 8 ml
Automatic switch-off:	after approx. 30 minutes resp. when tank is empty
Operating conditions:	+10°C - +40°C, relative humidity, 15-93 % -20°C - +70°C, relative humidity ≤ 93 %
Storage conditions:	98g (without batteries)
Weight appr.:	72 x 40 x 113 mm
Dimensions LxWxH appr.:	48552
Article no.:	4015588 48552 7
EAN-Number:	Art. 54116 Nebulizer unit (Parts no. /)
Spare parts:	Art. 54117 Mask set with mouthpiece Art. 54119 Mains adaptor

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

#### Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

#### The following warranty terms apply:

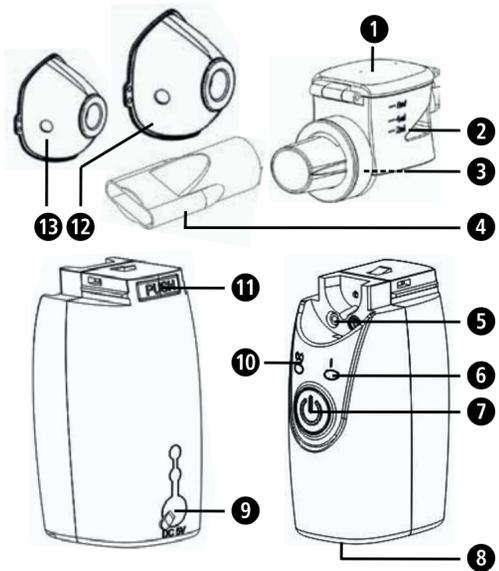
- The warranty period for **medisana** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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#### medisana GmbH