


### Device and controls

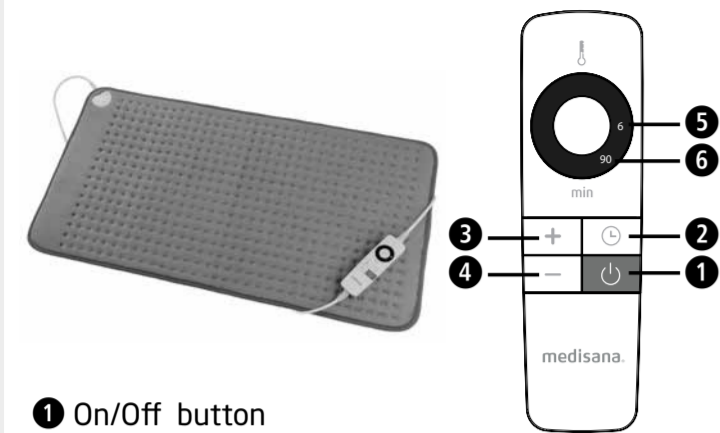

- 1 On/Off button
- 2 Timer button
- 3 + button
- 4 - button
- 5 Temperature level - orange LED indicator
- 6 Timer - blue LED indicator

### Explanation of symbols

- Do not use the heat pad when it is folded!
- Do not puncture the heat pad
- Not suitable for children under 3 years!
- Only use the heat pad indoors!
- The heat pad can be washed by hand! You may wash the cushion cover in a normal wash at max. 30 °C!
- Do not bleach!
- Do not dry the heat pad in a tumble drier!
- Do not iron the heat pad!
- Do not dry clean!

**IMPORTANT!**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes provide useful additional information on the installation or operation.

Protection category II

LOT number

Manufacturer



### GB Safety instructions

## IMPORTANT INSTRUCTIONS. RETAIN FOR FUTURE USE.



#### Power supply

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- The control switch should not be covered or placed on or below the pad while the unit is in operation.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or turn the heat pad by the mains lead and do not let the lead get tangled.
- Never touch a device that has fallen into water. Unplug the unit from the mains outlet immediately.
- The controls and leads must not be exposed to any kind of moisture.
- The heat pad may only be operated with the control unit (HP 720) supplied.

#### For special people

- Do not use the heat pad on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The heat pad must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians and unless the child has been well instructed how to use the control unit properly.
- Should you have any concerns about health issues, consult your doctor before using the heat pad.
- Do not use the heat pad on parts of the body that are swollen, injured or inflamed.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you wear a pacemaker, please be aware that the electromagnetic fields emanating from this electric product may, under certain circumstances, interfere with the functions of your pacemaker. Therefore, ask your doctor and the manufacturer of the pacemaker for advice before using this product.
- If you experience any pain or discomfort while using the unit, stop using it immediately.
- The appliance can be used by children older than 3 years and younger than 8 years under supervision and with the control always set to minimum temperature value.
- The cable and control unit of the appliance can give rise to risks of entanglement, strangulation, tripping or treading if not correctly arranged. The user shall make sure that excess ties and electric cords shall be arranged in a safe way.

#### Operating the device

- Only use the device for its intended use as in the instruction manual.
- If misused, all guarantee rights shall become null and void.
- Carefully check the heat pad before each use, if you notice wear or damage.
- Do not use if you notice wear, damage or signs of improper use on the heat pad, the switch or the cables or if the device is not working.
- Do not fold or crease the heat pad during use.
- Neither pierce the pad with, nor attach safety pins or other spiky or sharp objects to the pad.
- Never use the pad unsupervised.
- The device is only for private use and is not to be used in hospitals.
- Do not fall asleep while the heat pad is switched on. Prolonged use of the heat pad at a high setting could lead to skin burns.
- Do not cover the pad with any other cushions.
- Do not sit on the heat pad. Instead, rest the pad on or against the part of the body to be treated.
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).

#### Maintenance and cleaning

- You are only permitted to clean the unit. To avoid hazards, never repair the device yourself. Contact the customer service.
- Cleaning and user maintenance shall not be made by children without supervision.
- If the power supply cable is damaged, it may only be replaced by medisana, an authorised dealer or by qualified personnel in order to avoid hazards.
- A damaged power supply cable must be replaced with a power cable of the same type.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- When storing the heat pad, allow it to cool down before laying out flat in a clean and dry place.
- Do not store the device with any objects covering or resting on top of it in order to avoid sharp crinkles.

#### Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the it and contact your dealer or your service centre.

The following parts are included:

- 1 medisana Heat pad XXL HP 720
- 1 Detachable controller
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**  
Please ensure that the polythene packing is kept away from the reach of children!  
Risk of suffocation!

#### Operating

Press On/Off button 1 on the controller. The orange LED indicator 5 and the blue LED indicator 6 light up. After a few minutes, you will be able to feel the heat pad warming significantly. If you need a higher temperature setting, press the + button 3 to switch to the next temperature level, or to the maximum temperature level-6. If the heat pad is too hot for you, please press the - button 4 to return the heat pad to the 2nd or 1st step. The heat pad will cool down after a while.

Please set the heat pad to level 1 before falling asleep or using it for a long time.

Press Timer button 2 to adjust the automatic switch-off time. The automatic switch-off time is 30, 60, 90 minutes selectable. If you want to continue using the heat pad after the automatic switch-off, please press the On/Off button 1 and then repeat the steps above to select temperature level and timer.

Turn off the heat pad by pressing the On/Off button 1 and unplug when it is not in use.

#### Troubleshooting

If there is no function at all after switching on the heat pad (Orange and Blue LED indicator 5 6 shows no light), please check if the mains plug is correctly plugged in the mains outlet. If this is the case, the heat pad is defect. Please do not use the heat pad and contact the service center.

#### Cleaning and maintenance

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heat pad is equipped with a removable control unit.
- Unplug the control unit from the heat pad by pulling out the connection cable from the plug on the heat pad.
- The heat pad can be cleaned dry (with a soft brush) or as described in the instructions on the product label.
- Never use strong detergents or hard brushes.
- To dry, lay the pad flat on a moisture-absorbent towel or mat.
- Only use the blanket again once it is completely dry.
- Reconnect the control unit to the heat pad in the correct manner.
- You should untangle the cable from time to time.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.

#### Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

#### Technical specifications

Name and model:	<b>medisana Super Cozy Fleece Heat Pad XXL HP 720</b>
Power supply:	220-240V~ 50/60 Hz
Power output:	100W
Automatic switch-off:	after approx. 90 minutes
Dimensions approx.:	Approx. 45 x 85 cm
Weight approx.:	Approx. 0.75 kg
Operating conditions:	Only use in dry rooms according to this user manual
Storage conditions:	Laid out flat, clean and dry
Art. number:	48832
EAN number:	40155 88 48832 0

**In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.**

#### Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.