
medisana®

GB Shiatsu Massage Seat Cover MC 826
TR Shiatsu Masaj Minderi MC 826

Art. No.: 88926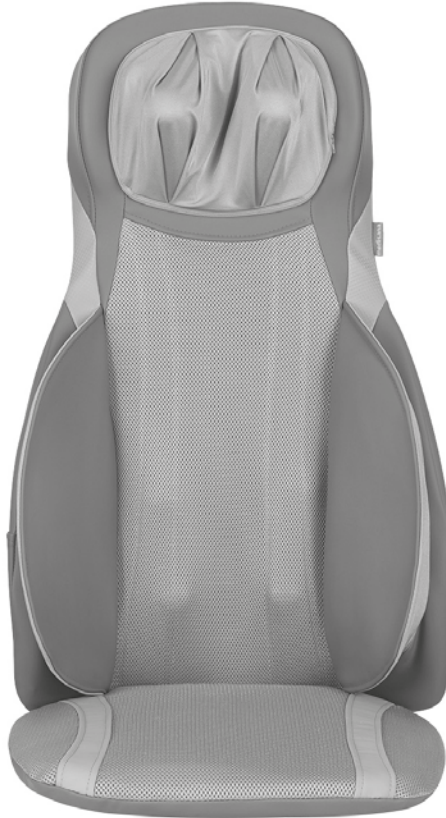

Certified Quality
Kalite Sertifikalı

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

GB Instruction Manual

1 Safety Information.....	4
2 Useful Information.....	5
3 Operating.....	6
4 Miscellaneous.....	11
5 Warranty.....	13

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Please fold out this page and leave it folded out for quick reference.

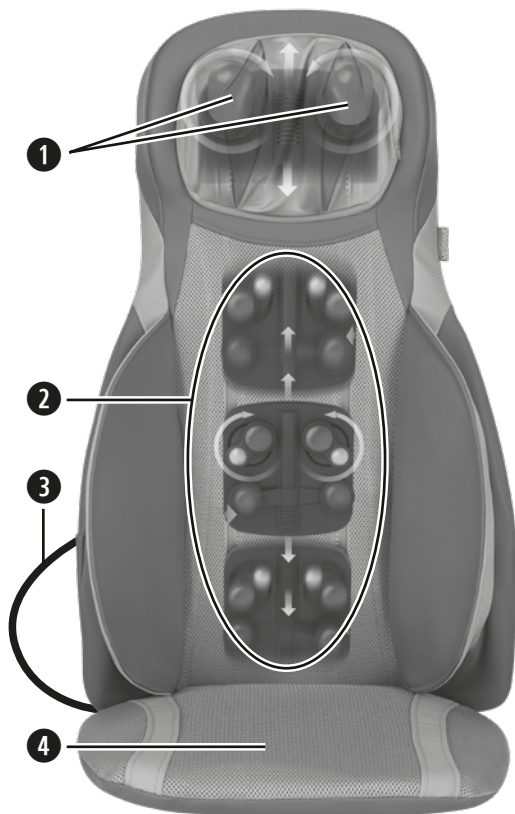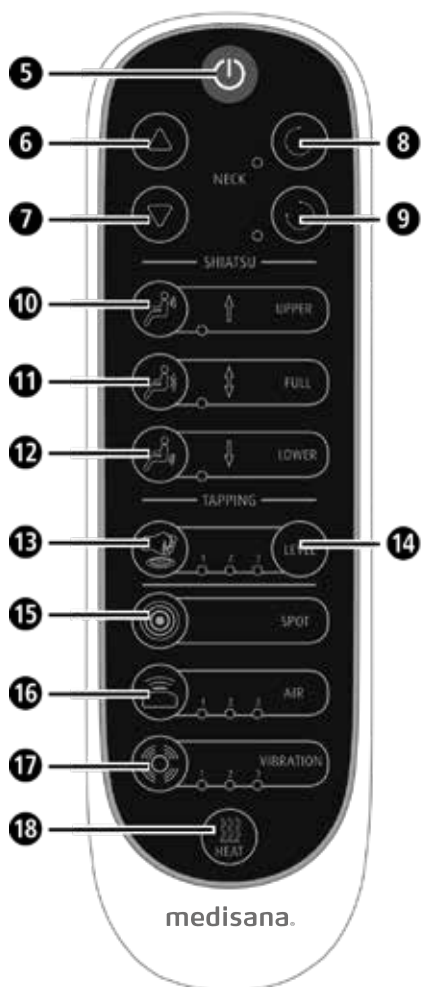- GB**
- ① 2 massage heads for neck massage
 - ② 4 rotating massage heads for back area
 - ③ Connection cable
 - ④ 2 vibrating massage motors in the upper thigh area
 - ⑤ On/Off button
 - ⑥ Massage head adjustment upwards for neck massage
 - ⑦ Massage head adjustment downwards for neck massage
 - ⑧ Neck massage with right rotation with LED
 - ⑨ Neck massage with left rotation with LED
 - ⑩ Button/LED for “Upper Back” massage
 - ⑪ Button/LED for “Full Back” massage
 - ⑫ Button/LED for “Lower Back” massage
 - ⑬ Button/Mode adjustment for tapping massage
 - ⑭ Button/3 LEDs for intensity setting of tapping massage
 - ⑮ Button spot massage (only with tapping massage)
 - ⑯ Button/3 LEDs for air massage
 - ⑰ Button/3 LEDs for vibration massage
 - ⑱ Button for heat function

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IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation and operation.

Protection category II

LOT LOT number

Manufacturer

about the power supply

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only connect the mains plug to the wall socket when the device is switched off.
- Do not operate this unit near high frequency electromagnetic sources.
- Keep the power cord and the device itself away from heat, hot surfaces, damp and liquids. Avoid touching the plug or power switch with wet or even damp hands, or when you are standing in water.
- Parts that conduct electricity on the device must not come into contact with liquid.
- If the device falls into water, do not try to pick it up. Instead pull the plug out immediately.
- Always make sure that the plug is easily accessible when the device is connected.
- Disconnect the plug from the socket immediately after using the device.
- Never pull on the power cord to disconnect the appliance from the mains. Always pull out the mains plug!
- Do not carry, pull or rotate the device by its power cord.
- Ensure that the cable will not cause tripping. The cable must not be kinked, trapped or twisted.

specific groups of people

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- The surface of the appliance is hot. People who are insensitive to heat must be careful when using the appliance.

- Do not use this unit to supplement or replace medical treatment. Chronic complaints and symptoms may be made worse.
- You should refrain from using the **Shiatsu Massage Seat Cover MC 826**, or should consult your doctor before using it, if
 - you are pregnant,
 - you have a cardiac pacemaker, artificial joints or electronic implants,
 - you suffer from one or more of the following illnesses and complaints: Circulation disorders, varicose veins, open wounds, bruises, broken skin, inflammation of the veins.
- If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.

before using the unit

- Always carefully inspect the cable, the control unit and the massage cushion for damage before use. Do not attempt to use a defective unit.
- Do not use the unit if there are visible signs of damage on the unit or the cable components, the unit does not work properly or the seat or the control unit have been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.

operating the unit

- The massage seat cover must only be used for its intended purpose as described in the instruction manual.
- Using the seat cover for any other purpose invalidates the warranty.
- Never leave the unit unattended if it is connected to the power supply.
- Do not allow the massage seat cover to come into contact with pointed or sharp objects.
- Never put the unit down next to an electric stove or other sources of heat.
- Do not stand on the unit.

- Operate the device in an upright position only, never place it flat on the floor.
- If you feel unwell or experience pain when using the unit, stop the treatment immediately.

The device is only intended for household use and must not be used for commercial or medical purposes. If you have health concerns, consult your doctor before using the Shiatsu massage seat cover.

The massage seat cover must only be used in enclosed spaces!

Do not use the massage seat cover in damp rooms (e.g. when bathing or showering).

maintenance and cleaning

- The unit is maintenance free.
- Do not attempt to repair the unit yourself in the event of any defects. Not only does this render the warranty null and void but you can also run the risk of serious hazards (fire, electric shock, injury). Only have repairs carried out by authorised service points.
- Cleaning and user maintenance shall not be made by children without supervision.

Do not wash!

Do not dry clean!

Thank you very much

Thank you for your confidence in us and congratulations!

You have acquired a **MEDISANA** quality product with this **Shiatsu Massage Seat Cover MC 826**.

This unit is designed for massaging the back, neck and upper thigh areas. We recommend that you carefully read the following information on use and care of the product so that you experience its benefits and enjoy using your **MEDISANA Shiatsu Massage Seat Cover MC 826** for many years.

**2.1
Items
supplied
and
packaging**

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your supplier or your service centre.

The following parts are included:

- 1 **MEDISANA Shiatsu Massage Seat Cover MC 826** with control unit
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your supplier without delay.

**WARNING**

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

**2.2
What is
Shiatsu
massage?**

Shiatsu is a form of finger pressure massage and one of the most important massaging techniques along with manual lymphatic drainage and reflexology.

This form of physical therapy was developed in Japan but stems from traditional Chinese massage. Gentle stroking and beneficial pressure application techniques are used to awaken life forces and encourage them to flow through the body.

The object of the massage is not just to ease tension, but also to optimise the flow of life force and harmonise body and soul.

2.3 How does the MEDISANA Shiatsu Massage Seat Cover work? In the **MEDISANA Shiatsu Massage Seat Cover MC 826** you have a modern device for genuine Shiatsu or tapping style back massages in the back area. The **MEDISANA Shiatsu Massage Seat Cover MC 826** has 4 rotating massage heads for Shiatsu massage of the three zones “Upper Back”, “Lower Back” and “Full Back” as well as 2 massage heads for the neck area. The device also offers the possibility of an air massage (with 3 different intensities) for the lumbar area. The device is equipped with an additional 3-level vibration function and a red light heat function in the massage heads. The unique combination of intensive shiatsu and vibration massage with heat makes the massage seat cover an effective and relaxing treatment. Their settings can be comfortably adjusted by means of the easy-to-handle control unit.

3 Operating

3.1 Usage The use of the Shiatsu Massage Seat Cover will improve your well-being after a stressful day. The massage effect is particularly useful for tense muscles and tired tissue. The unit provides pleasant relaxation for overcoming everyday stress.

3.2 Operation **Start-up**

- Place the massage seat cover on a high chair or armchair and attach it to this by means of the fasteners on the back.
- Insert plug into the mains socket. Ensure that the power supply is easily accessible.
- Take a seat on the Shiatsu massage seat cover and familiarize yourself with the functions.

Operating the unit

The **Shiatsu Massage Seat Cover MC 826** offers a soothing massage for back, neck and thighs as well as an air massage for the lumbar area. The additional selectable heat function can only be used together with the Shiatsu or tapping massage, during which the vibration massage (with 3 selectable intensity levels) can also be activated. Do not use the unit for more than 15 minutes at a time. The unit will switch off automatically after max. approx. 15 minutes. Always allow the unit to cool down fully before you use it again.

3.2

Operation

Use the buttons on control pad to operate the massage seat cover:

- Switch on the device by pressing On/Off button **5**. The power LED illuminates.
- The buttons **6**, **7**, **8** and **9** control the neck massage. You may adjust the position and the rotation direction of the both massage heads. The LEDs next to the buttons **8** and **9** indicate, which rotating direction is active.
- The buttons **10**, **11**, **12**, **13**, **14** and **15** allow a Shiatsu- or tapping massage for the back area. With the button **15** you may activate a spot massage (only with tapping massage).
- When the “Upper Back” massage button **10** is pressed, the LED next to the button illuminates and the Shiatsu massage heads **2** move from the “Park” position at the base of the backrest, up to the shoulder area, massaging the back in a circular motion whilst doing so. Once the uppermost position has been reached, the direction of rotation of the massaging heads is reversed. The heads then travel down to the centre of the back. Once the heads reach this point, the direction of rotation is reversed again and then travel up the back again. If you press button **10** during the massage, this function is switched off and the massage heads remain stationary in their current position.
- If you press the “Lower Back” massage button **12**, the LED next to the button illuminates. The Shiatsu massage heads **2** travel up and down the lower half of the back in a circular motion. The reversal of the direction of rotation takes place when the respective end positions are reached. Pressing button **12** during the massage, switches off this function and the massage heads remain stationary in their current position.
- If you press the “Full Back” massage button **11** the function procedure is similar to pressing buttons **10** and **12**, except that it is for the entire back.
- To activate a tapping massage for the selected back area, press the button **13**. By repeatedly pressing this button, you can change the tapping massage mode (mode 1 = constant massage, mode 2 = massage with pauses, mode 3 = alternating mode 1 and mode 2 with change of moving direction). If mode 3 is active and you press the button **13** again, the tapping massage will be switched off.
- The button **14** (level) controls the intensity of the tapping massage. After switching on the tapping massage the LED “1” next to the button lights up (1 = low intensity). A press on the button **14** increases the intensity (2 = medium intensity). A second press on the button **14** increases the intensity again (3 = high intensity). After button is pressed a third time, the tapping massage intensity is set back to intensity 1.
- During the tapping massage, press button **15** to activate spot massage.

3.2 Operation

- You can enhance the Shiatsu massage at any time with a vibration massage for your thighs by pressing the vibration massage button ⑰.
- The vibration massage in the upper thigh area can also be operated separately by pressing vibration massage button ⑰. Level indicator LED “1” next to the button lights up and both vibration massage motors are switched on. They begin with a gentle massage. Press button again and the vibration level increases (medium intensity massage LED “2”). Press button a third time and the maximum vibration level (highest intensity massage, LED “3”) is switched on. After button is pressed a fourth time, the vibration massage is switched off and LED “3” next to the button goes out.
- To switch on the air massage for the lumbar area press button ⑱. The LED “1” next to the button lights up (1 = low intensity). A second press on the button ⑱ increases the intensity (2 = medium intensity). A third press on the button ⑱ increases the intensity again (3 = high intensity). After button is pressed a fourth time, the air massage is switched off.
- The heat function can be activated at any time with the Shiatsu or tapping massage, during which the vibration massage can also be activated. Press the heat button ⑲ to activate the function; the LED lights up. The heat is radiated through the massage heads ②. Press the button ⑲ again to de-activate the heat function; the LED goes out.
- *To switch off the device:* Press the On/Off button ⑤. The power LED begins to flash and the massage heads travel back down to the “Park” position. When this position is reached, the LED extinguishes. If you press the On/Off button ⑤ when one of the massage functions is switched on, the vibration motors just stop immediately. First the massage heads travel downwards to the “Park” position. The power LED flashes meanwhile and then extinguishes.
- After each use, switch the device off using the On/Off switch ⑤ and remove the plug from the mains socket.

4 Miscellaneous

4.1 Cleaning and main- tenance

- Before cleaning the device, make sure that it is turned off and the mains plug is removed from the socket. Allow the unit to cool down.
- Only clean the Shiatsu massage seat cover with a soft cloth slightly dampened with a mild detergent (e.g. soap suds). Under no circumstances should you use aggressive detergents, strong brushes, solvents or alcohol.
- When cleaning, never immerse the unit in water and make sure that no water gets into the unit.

4.1
Cleaning
and
main-
tenance

- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

4.2
Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

4.3
Technical
Specifica-
tions

Name and model	: MEDISANA Shiatsu Massage Seat Cover MC 826
Voltage supply	: 220-240V~ 50/60Hz
Power consumption	: approx. 72 W
Automatic switch off	: after approx. 15 minutes
Operating conditions	: only use in dry rooms
Storage conditions	: in a cool, dry place
	Storage temperature 0 °C - max. 40 °C
Dimensions	: approx. 80.5 x 48 x 22.5 cm
Weight	: approx. 8.4 kg
Article No.	: 88926
EAN Code	: 40 15588 88926 4

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.

**Warranty
and
repair
terms**

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is two years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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Zhangzhou Easepal Industrial Co., Ltd