Explanation of symbols

IMPORTANT
Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

Only use the device indoors.

Do not use the device in damp rooms (e.g. when bathing or showering).

Protection category II

LOT Lot number

GB Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

The device must only be used for its intended purpose as described in the instruction manual. Using the appliance for any other purpose invalidates the warranty.

The device is not to be used for commercial use or medical purposes.

about the power supply

- Please fully charged the battery for first time use to the optimize the battery usage.
- Battery fully charged time approximately ≤4h
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only insert the power supply unit into a socket when the appliance is switched off.
- Only use the power supply unit supplied for the massager. It must only be supplied at extra low voltage corresponding to the marking on the appliance.
- Keep the power supply unit, cable and appliance well away from heat, hot surfaces, moisture and liquids.
- Do not touch the power supply unit if you are standing in water or if your hands are wet or damp.
- Do not take hold of a device if it falls into water. Pull out the power plug and the power supply unit from the socket immediately.
- Only plug the device into a socket where the plug is freely accessible.
- Always switch off the appliance after use with the buttons on the control unit and unplug the power supply unit.
- To disconnect the unit from the power supply, always pull the plug out of the socket. Do not pull on the cable!
- Do not carry, pull or twist the device by the cable or the power supply unit.
- You must not continue to use the unit if the cable or power supply unit are damaged. For safety reasons, these parts may only be replaced by authorized service personnel. In order to prevent hazards, always send the unit in to the service centre.
- Ensure that no one can trip over the cable. The cable must not be kinked, trapped or twisted.

specific groups of people

- The device can be used by children aged from 15 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Do not use this unit to supplement or replace medical treatment. Chronic complaints and symptoms may be made worse.
- Do not use the massager, or only after consulting your doctor, if:
 - you are pregnant,
 - you have a cardiac pacemaker, artificial joints or electronic implants,
 - you suffer from one or more of the following illnesses and complaints: Circulation disorders, varicose veins, open wounds, bruises, broken skin, inflammation of the veins.

- Do not use the appliance near your eyes or near any other sensitive areas.
- If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.
- Take care when using the heating function. The appliance has hot surfaces. Do not use the appliance on infants, vulnerable individuals or people who are not sensitive to heat.
- Before using the massager, consult your doctor regarding any unexplained pain, if you are undergoing medical treatment or using medical equipment.

before using the unit

- Check the power supply unit, cable and massager carefully for damage before each use. Do not attempt to use a defective unit.
- Do not use the appliance if it, or the power cord show signs of damage, if it is not in perfect working order, or if it has been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.
- Do not keep or place any electrical device on a surface where it is in danger of falling into a bathtub or sink.
- Only use the appliance in the proper place, as described in this instruction manual.
- Do not use the appliance while you are asleep or when you are lying in bed.
- Do not use the appliance before going to sleep. The massage has a stimulating effect.
- Never use the appliance while driving a vehicle or operating a machine.
- The maximum operating time for one session is 15 minutes. Longer use shortens the life of the device and can lead to tension of the muscles instead of relaxation.
- Very long periods of use can lead to overheating. Always allow the appliance to cool down before using it again.
- Never store or use the appliance next to an electric stove or other sources of heat.
- Never leave the unit unattended if it is connected to the power supply.
- Do not allow the device to come into contact with pointed or sharp objects.

- Do not cover up the appliance when it is switched on. Never use it under blankets or cushions. There is a risk of fire, electric shock or injury.
- Make sure, that no fingers or other parts of the body come between the rotating massage heads. Risk of injuries!
- Do not use an external time switch or a separate remote control system to operate the appliance.
- Do not use the device whilst using aerosol sprays or simultaneously using oxygen.

Intended use

The device is used to massage certain body parts for relaxation and well being. It is not intended for medical therapeutic use. It is not suitable for clinics, massage practices, wellness studios etc. The massage device is not a medical device and is not intended for the treatment of pain. In case of doubt or uncertainty, consult your doctor before use.

WARNING
Please ensure that polythene packing is kept away from the reach of children! Risk of suffocation!

Device and controls

- Warmth-button (switches warming function on or off)
- Massage level of intensity button
- Mode setting button (massage head clockwise or anti-clockwise)
- ON/OFF button
- Massage heads
- Connection for power supply

Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre. The following parts are included:

- 1 **medisana** Wireless 6-in-1 Neck Massager NM 900
- 1 Power supply
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your dealer without delay.

Operation


- Insert the power supply unit into the connection for power supply 6 on the backside of the device. Plug in the power supply unit to charge battery
- Please fully charge the battery for first time use to the optimize the battery usage.
- Battery fully charge time approximately ≤4h
- Place the massage device around the neck with the control panel facing upwards in such a way it receives sufficient support on the shoulders
- Please see illustration to use hand straps:

- On/Off button 4
 - Press the On/Off button for approximately 1 second to turn on the device, the button then shows green lighting
 - The device is default to start as: Massage head rotates anti-clockwise, level 2 of massage intensity, no warming function
 - Press the On/Off button for approximately 2-3 seconds to turn off the device
 - After 15 minutes of constant massage the device will turn off automatically and massage heads will resume to original position
- Massage level of intensity button 2:
 - Level 1: Low intensity, shows 1 lighting
 - Level 2: Medium intensity, shows 2 lightings
 - Level 3: High intensity, shows 3 lightings
- Mode setting button 3:
 - Press the mode setting button to switch the massage heads rotation direction to clockwise or anti-clockwise. The lighting runs clockwise or anti-clockwise according to the massage heads rotation direction
- Warmth-button 1:
 - Press the warmth-button to turn on the warming function, the button then shows red lighting
 - To turn off the warming function press the warmth-function button again
- Low battery indication 4:
 - When battery is lower than 15%, the On/Off button shows flashing red lighting as a reminder for battery recharging
 - When charging battery, it shows red lighting flash slowly
 - After battery fully charged, it shows green lighting

Cleaning and maintenance

- Cleaning and user maintenance shall not be carried out by children without supervision.
- If a fault does occur, just check whether the mains power adapter is properly connected. Do not open the device. There are no parts in it that are to be cleaned or serviced by the user.
- You may only clean the device yourself. If the device malfunctions please do not repair it yourself as this leads to the automatic expiry of all warranty rights. Consult your specialist dealer and have repairs carried out by the service centre.
- Before cleaning the appliance, make sure that the appliance is switched off. Allow the unit to cool down.
- Clean the massager with a slightly damp sponge only. Never use brushes, aggressive cleaning agents, petrol, thinners or alcohol. Rub the massage dry with a clean cloth. Do not dry on a radiator or with the hair dryer.
- The cover of the massage heads can be removed for cleaning. The cover can be cleaned according to the care instructions on the label. Please see below illustration for how to take off the cover for cleaning.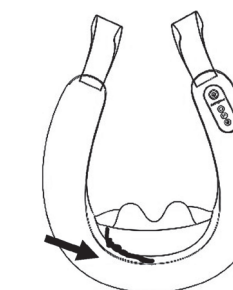

- Never immerse the device in water to clean it and ensure water cannot get into the device.
- If, nonetheless, liquid should penetrate the device, unplug the mains cable immediately.
- Wait until the device is completely dry before using it again.
- Take care when rolling up the mains lead to avoid damaging it.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

Disposal

The battery in the appliance contains environmentally hazardous substances. Before the appliance is discarded, the battery must be removed, and the power should be cuff-off before the battery removed. Then the battery should be discarded in a safe situation.

This product must not be disposed together with domestic waste all users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at the municipal or commercial collection point so that they can be disposed in an environmentally acceptable manner.

Consult your municipal authority or your dealer for information about disposal.

Technical specifications

Name and model : **medisana** Wireless 6-in-1 Neck Massager **NM 900**
 Battery specifications : 11.1V 2200mAh (3 batteries)
 Input: 100-240V, 50-60Hz
 Power supply: Output: DC 14V = 1.7A
 Dimensions LxWxH : Approx. 398 x 340 x 170 mm
 Weight : Approx. 2.1 kg (with power supply)
 Storage conditions: Clean and dry
 Item number : 48895
 EAN number : 40 15588 48895 5

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for medisana products is 1 year from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
 - All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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備考1. “超出0.1 wt %”及“超出0.01 wt %”係指限用物質之百分比含量超出百分比含量基準值。
備考2. “○”係指該項限用物質之百分比含量未超出百分比含量基準值。
備考3. “—”係指該項限用物質為排除項目。
Note 1: “Exceeding 0.1 wt %” and “exceeding 0.01 wt %” indicate that the percentage content of the restricted substance exceeds the reference percentage value of presence condition.
Note 2: “○” indicates that the percentage content of the restricted substance does not exceed the percentage of reference value of presence.
Note 3: The “—” indicates that the restricted substance corresponds to the exemption.

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