
Explanation of symbols

IMPORTANT
Please read the instructions carefully and follow the directions before use.
Non-observance of these instructions can result in serious injury or damage to the device.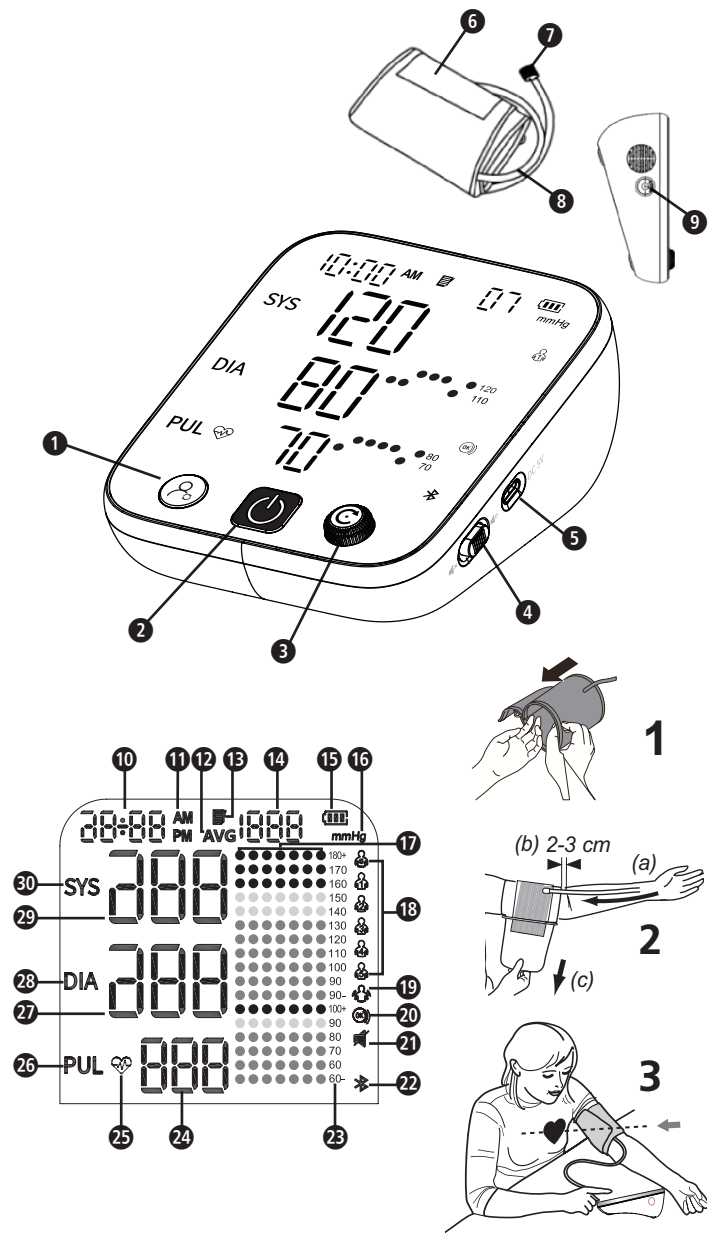

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

Device classification: type BF applied part

Temperature range

IP21 Ingress protection rating

RoHS RoHS mark

LOT LOT number **SN** Serial number

Manufacturer  Date of manufacture

Device information:
• Internally powered equipment
• Not suitable for use in presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide

Intended use

- This fully automatic over-the-counter electronic blood pressure monitor is intended for measuring human's systolic and diastolic blood pressure and heart rate by using an oscillometric technique by means of a cuff, which needs to be fitted on the upper arm. All values can be read out in one LED panel. The device is intended for adult indoor use at home only.
- The device is able to detect irregular heartbeat and indicate this with a symbol.

Contraindication

- The device is not suitable for use on pregnant women and patients with implanted, electronical devices (e.g. pacemaker or defibrillator).

GB Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
- Only use the unit for its intended purpose in accordance with the instructions for use.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.
- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- Consult your physician to get more information about your blood pressure. A self-diagnosis and self-treatment based on the measured values may be dangerous. Always follow the instructions given by your physician!
- If you experience discomfort during a measurement, such as pain in the upper arm or other complaints, press the Start / Stop Button **2** to immediately to stop the measurement, loosen the cuff and remove it from your arm.
- The device only works correct with the fitting cuff.
- The unit is not suitable for children.
- Do not allow children to use the unit. Medical products are not toys!
- Please ensure that the unit is kept away from the reach of children.
- Do not put the air tube around your neck - danger of suffocation!
- The swallowing of small parts like packaging bag, battery, battery cover and so on may cause suffocation.
- The user needs to check that the equipment functions properly and that it is in proper working condition before being used.
- Only the cuff provided can be used, and cannot be changed to any other branded cuff. It can only be replaced with a cuff of exactly the same type.
- The device may not be operated in rooms with high levels of radiation or in the vicinity of high-radiation devices, such as radio transmitters, mobile telephones or microwaves, as this could cause functional defects or incorrect measurements.
- Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquids (such as alcohol) are present.
- No modification of this unit/device is allowed.
- Please do not attempt to repair the unit yourself in the event of malfunctions. Only have repairs carried out by authorized service centers.
- Protect the unit against moisture. If liquid does manage to penetrate into the unit, remove the batteries immediately and do not continue to use the unit. In this case contacts your specialist dealer or notify us directly.
- Please do not use a dilution agent, alcohol or petrol to clean the unit.
- Please do not hit the device heavily or let it fall down from a high place.
- Do not use mobile phones in the range of the device. This may lead to malfunctions.
- If the unit is not going to be used for a long period, please remove the batteries.

BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Replace all batteries if the unit display shows the low battery symbol.
- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed, seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period of 3 months or more!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Unit and LED-Display

- | | | |
|--|--|--|
| 1 User Selection / Setting Button | 2 START / STOP Button | 3 Memory Button |
| 4 Voice ON/OFF Button | 5 USB-C Power Supply Port | 6 Cuff |
| 7 Connector for the Hose | 8 Air Hose | 9 Air Jack |
| 10 Date and Time | 11 AM / PM Icon | 12 Average Value / Triple Measurement Mode Icon |
| 13 Memory Icon | 14 Memory Number | 15 Battery Icon |
| 16 Unit of Blood Pressure | 17 Weekly Blood Pressure Log | 18 User Icon |
| 19 Movement Detection Icon | 20 Cuff Fitness Detection Icon | 21 Mute / Unmute Icon |
| 22 Bluetooth Icon | 23 Blood Pressure Segments (mmHg) | 24 Pulse Rate |
| 25 Heartbeat Icon | 26 Pulse Rate Icon | 27 Diastolic Pressure Value |
| 28 Diastolic Pressure Icon | 29 Systolic Pressure Value | 30 Systolic Pressure Icon |

Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre.
The following parts are included:
• 1 **medisana** blood pressure monitor
• 4 batteries (AA, LR6) 1.5V
• 1 instruction manual
• 1 cuff with air tube
• 1 USB-A to USB-C cable
Please contact your supplier if you notice any transport damage on unpacking the unit.

WARNING - Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

How is blood pressure measured?

The **medisana** is a blood pressure unit which is used to measure blood pressure at the upper arm. The measurement is carried out by a microprocessor, which, via a pressure sensor, measures the vibrations resulting from the inflation and deflation of the cuff over the artery.

Blood pressure classification

systolic mmHg	diastolic mmHg	Blood pressure indicator
≥ 160	≥ 100	red
140 - 159	90 - 99	yellow
90 - 139	60 - 89	green
< 90	< 60	green

WARNING - Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least five minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least five minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Getting started - insert/ replace batteries


You must insert the batteries provided before you can use your unit. The lid of the battery compartment is located on the underside of the unit. Open it by slightly pushing it and moving it upwards. Remove it and insert the 4 AA batteries supplied. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment). Close the battery compartment. Replace the batteries if the low battery symbol appears on the display or if nothing appears on the display after the unit has been switched on.

USB Type-C for power supply

In addition to the batteries, the power can also be supplied by plugging into D.C. 5V externa power supply through a USB-C Power Supply Port **5**. (The USB-C cable is included in the packing)

Settings - date, time and unitWhen the device is turned off, press the "R" Setting button **1** for about 3 seconds to enter the date setting interface, and the "year" will flash. Rotate the "C" Memory button **3** to adjust the year, and press the "R" Setting button **1** to confirm the selection. When the year is set, it will automatically enter the month setting. Rotate the "C" Memory button **3** to adjust the month, and press the "R" Setting button **1** to confirm the selection. Follow the same steps to adjust the date/hour/minute.

When the time is set, it will automatically enter the measurement unit setting. Rotate the "C" Memory button **3** to change the measurement unit to mmHg / kPa, and press the "R" Setting button **1** to confirm the setting.

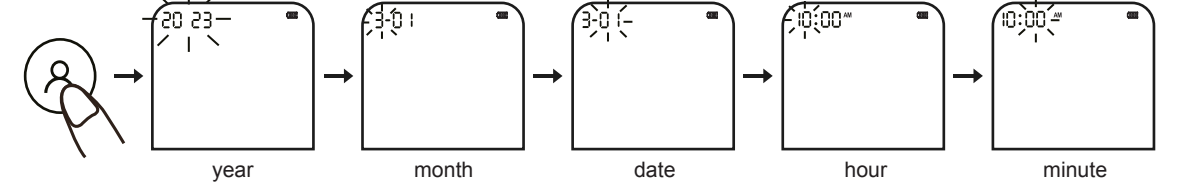Voice setting

Use the Voice ON/OFF button **4** on the side of the device to activate or deactivate the voice output.

Fitting the cuff

- Connect the air hose **3** to the air jack **9** on the left side of the device.
- Slide the cuff over your left upper arm (Fig.1).
- Position the air hose in the middle of your arm in line with your middle finger (Fig.2) (a). The lower edge of the cuff should be 2 - 3 cm above the crease of the elbow. (b). Pull the cuff tight and close the Velcro fastener (c).
- Tighten the end of the cuff so that it wraps evenly around your arm, leaving room for 2 fingers between the cuff and your arm.
- Measure the blood pressure on your bare arm.
- Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.
- Correct measuring position for sitting (Fig.3). Make sure that the cuff does not slip during measurement and that the printed arrow points to the crook of the arm.

User memory setting

The **medisana** blood pressure monitor features 1000 memories for each of 5 users and a guest mode (👤) (There is no memory for guest user). Press the "R" user selection button **1** to select the desired user memory **10** (👤-👤).

Taking a blood pressure measurement

After the cuff has been appropriately positioned, the measurement can begin:

- Switch the unit on by pressing the START/STOP button **2**.
- The number 00 flashes for 3 seconds. The device then automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display. The Cuff Fitness Detection icon **20** is always displayed on the screen when the cuff is worn correctly. When the cuff is worn too loosely, the icon will flash to remind you. The Movement Detection Icon **19** flashes when body movement is detected. It may cause inaccurate measurement results. Please restart the measurement.
- The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the Heartbeat icon **25** on the display starts to flash.
- When the measurement is finished the systolic and diastolic blood pressure and the pulse value appears on the display. The weekly blood pressure log **17** display relevant color depending on the blood pressure classification. If the unit has detected an irregular heartbeat, the Heartbeat icon **25** keep flashing. The device automatically turns off after 1 minutes.

Triple measurement mode

After the cuff has been appropriately positioned, the measurement can begin:

- When the device is power off, press and hold the START/STOP button **2** for 3 seconds to activate the triple measurement mode.
- The Triple measurement mode icon **12** flashes for 3 times and then the device takes the first measurement.
- After the first measurement is completed, the value is displayed for about 5 seconds, and then the device takes a 60-second break with a countdown display.
- Once the countdown ends, the second and third measurement begin with the same procedure as the first measurement.
- After all three measurements are completed, the device calculates the average blood pressure and pulse rate. The results are displayed on the screen.

Transfer via Bluetooth®

This device offers the possibility to transfer your measured values via Bluetooth® to the VitaDock® app. The VitaDock® app allows the evaluation, storage and synchronisation of your data between multiple iOS- and Android-devices to have access to it anytime and anywhere. You may share your results with your friends or your doctor. Therefore you need a free user account, which you can create on the website www.vitadock.com. For Android or iOS mobile devices, you may download the respective apps. You will find a detailed instruction for how to install and use the software on the mentioned website. After each measurement an automatic transfer of the values will take place (provided that Bluetooth® is activated and configured on the receiving device).

WARNING
• Keep the device at least 20 centimeters away from the human body (especially the head) when the Bluetooth® data transmission is proceeding after measurement.
• The distance between the device and the receiver should be between 1 meter to 10 meters to avoid possible disturbances.

Bluetooth® type: 4.2 BLE; System requirements: Android 6.0 or newer; iOS 7.0 or newer
RF Frequency range: 2402 MHz - 2480 MHz; Equivalent isotropic radiant power: 3,99 dBm;
Maximum permitted power: 0,47 mW; Transmitting distance: 10 m (Class 2); Data throughput: 1 Mbps

Displaying stored results

This unit features 5 separate memories, each with a capacity of 1000 memory logs. Results are automatically stored in the memory selected by the user. When a user memory reaches 900 logs, the pulse rate **24** will display a flashing "FUL". When the user memory reaches 1000 logs, the pulse rate **24** will display "END". There are two memory display modes:

- Daily average display mode
When the device is turned off, press the Memory button **3** to enter daily average display mode. The daily average value of the last day is displayed on the left, and the weekly blood pressure trend **17** of daily average value of the last seven days is displayed on the right. Rotate the Memory button **3** to view the average value of each day.
- Single value display mode
When the device is turned off, press and hold the Memory button **3** for about 3 seconds to enter single value display mode. The last measured value is displayed on the left, and the blood pressure trend **17** of the last seven measured values is displayed on the right. Rotate the Memory button **3** to view the measured value of each measurements.

Deleting stored values

If you are sure that you want to permanently remove all stored memories, press the "R" User selection button **1** in power-off mode to select the user memory whose measured values need to be deleted. Press the START/STOP button **2** to power off and press "C" Memory button **3** to activate the screen, then keep pressing the "C" Memory button **3** for about 3 seconds to delete the memories of the selected user and the icon "Trg" will be shown on the display.

Abnormal phenomena and handling troubleshooting

Abnormal displays	Cause	Solution
Er U	The cuff is too loose	Pleased try again
Er H	The inflation reaches 295mmHg, and it deflates automatically after 20ms	
Er 1	The pulse rate is not detected correctly	
Er 2	Too much disturbance (Move, talk, or magnetic disturbance during a measurement)	Move to a closer distance
Er 3	The pulse rate is too low (<40bpm) or too high (>199bpm).	
Er 23	SYS value reads lower than 57mmHg	
Er 24	SYS value reads higher than 255mmHg	
Er 25	DIA value reads lower than 25mmHg	
Er 26	DIA value reads higher than 190mmHg	
Failure to power on	Insufficient battery power	Replace the batteries or insert the USB Type-C cable for power supply
	Whether the positive and negative poles of the battery are installed reversely	Install the batteries correctly
No inflation	The air tube is not inserted correctly	Insert the air tube plug firmly into the jack
	The air tube is broken or leaking	Please contact the dealer to replace with a new cuff
Unable to measure due to the display error	Whether the arm is moved when pressurization	Keep your arm and body still
	Whether you talk during the measurement	Keep quiet while measuring the blood pressure
Air leakage of the cuff	Whether the cuff is twined too loose	Please tighten the cuff
	The airbag of the cuff is ripped	Please contact the dealer to replace with a new cuff

WARNING - If the blood pressure still cannot be measured after trying the above-stated solutions, please contact the dealer. Do NOT attempt to disassemble the device by yourself.

Cleaning and maintenance

Remove the batteries before cleaning the unit. Clean the unit using a soft cloth lightly moistened with a mild soapy solution. Never use abrasive cleaning agents, alcohol, naphta, thinner or gasoline etc. Never immerse the unit or any component in water. Be cautious not to get any moisture in the main unit. Do not wet the cuff or attempt to clean the cuff with water. Using a dry cloth, gently wipe away any excess moisture that may remain on the cuff. Lay the cuff flat in an unrolled position and allow the cuff to air dry. Do not expose the unit to direct sunlight; protect it against dirt and moisture. Do not expose the unit to extreme hot or cold temperatures. Keep the device in the storage case when not in use. Store the unit in a clean and dry location.

Disinfection

- If used by yourself in a home environment, wipe the cuff once a month with a commercially available cotton cloth with 75% alcohol for more than 30 seconds at a time for disinfection in addition to the normal cleaning routine.
- If used for more than one person (home use), wipe the cuff once a week with a commercially available cotton cloth with 75% alcohol for more than 30 seconds at a time for disinfection in addition to the normal cleaning routine.
- After cleaning/disinfection and before use, make sure that there are no blood stains or dirt on the display, the device or the cuff. If there are blood spots or dirt that cannot be removed, please dispose of the device without reusing it.
- If used in a complex environment (e.g. in a hospital) or by persons outside your own family, replace the cuff with a new one after use.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

Guidelines / Standards

This blood pressure monitor meets the requirements of the EU standard for noninvasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) "CE 0123 The blood pressure monitor corresponds to European regulations EN 60601-1 and EN 60601-1-2. The specifications of EU Guideline "MDR (EU) 2017/745" are met.

Technical specifications

Name and description:	medisana Upper arm blood pressure monitor (Model no.: ARM-HA101+)
Display system / Memory:	Digital display / 5 x 1000
Measuring method:	Oscillometric
Power supply:	4 x AA "LR6" Alkaline batteries 1.5V / Type-C DC 5V 1A
Blood pressure measuring range:	SYS: 57-255 mmHg DIA: 25-195 mmHg
Pulse measuring range:	40 - 199 beats per minute
Maximum error tolerance for static pressure:	± 3 mmHg
Maximum error tolerance for pulse rate:	± 5 % of the value
Pressure generation:	Automatic with pump
Deflation:	Automatic
Auto power-off:	1 minute after use
Battery life:	Approx. 100 measurements
Product life span:	Approx. 5 years with 4 times daily use
Operating conditions:	+5°C to +40°C, 15 to 90 % max. relative humidity
Storage / Transportation conditions:	-20°C to +55°C, 10 to 93% max. relative humidity
Dimensions / Cuff:	Approx. 120 x 140 x 63 mm / 22 - 42 cm
Weight:	Approx. 320 g without batteries
Item number:	50060
EAN number:	48971 38 50060 3

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:
1. The warranty period for medisana products is 5 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

Shenzhen AOJ Medical Technology Co., Ltd.
Room 301&4F, Block A, Building A, Jingfa Intelligent Manufacturing Park, Xiaweiyuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, China

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