
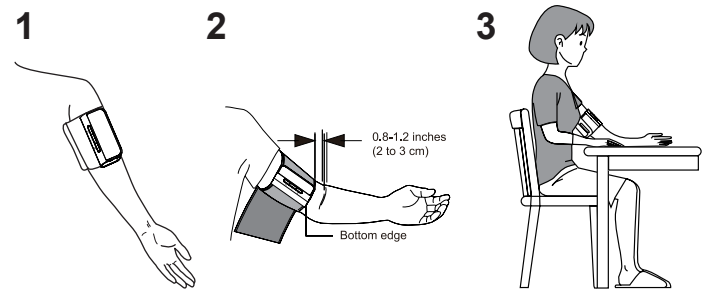
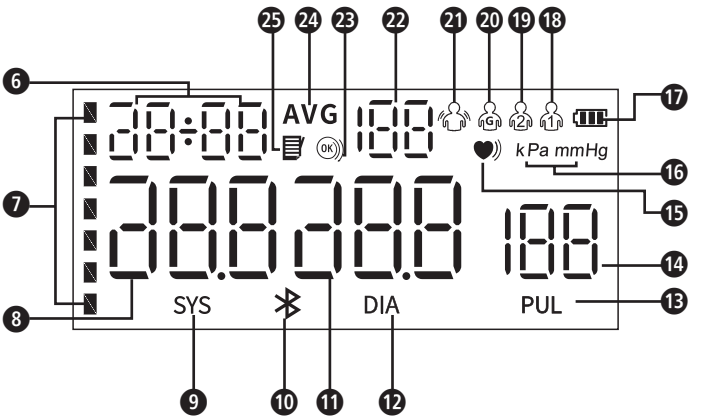
GB Instruction manual  
Upper Arm Blood Pressure Monitor  
BU 640 connect

Unit and Display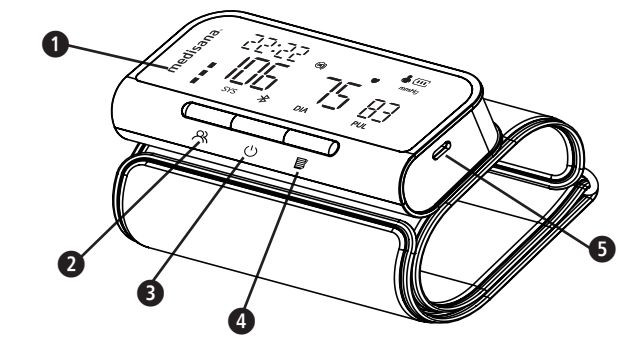

Explanation of symbols

**IMPORTANT**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes give you useful additional information on the installation or operation.

Device classification: type BF applied part

|             |                           |
|-------------|---------------------------|
| <b>IP22</b> | Ingress protection rating |
| <b>RoHS</b> | RoHS mark                 |
| <b>LOT</b>  | LOT number                |
| <b>SN</b>   | Serial number             |
| <b>MD</b>   | Medical device            |
| <b>UDI</b>  | Unique device identifier  |
|             | Importer                  |
|             | Manufacturer              |
|             | Date of manufacture       |

Device information:

- Internally powered equipment
- Not suitable for use in presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide
- Continuous operation with short-time loading

**Intended use**

- This fully automatic over-the-counter electronic blood pressure monitor is intended for measuring human's systolic and diastolic blood pressure and heart rate by using an oscillometric technique by means of a cuff, which needs to be fitted on the upper arm. All values can be read out in one LED panel. The device is intended for adult indoor use at home only.
- The device is able to detect irregular heartbeat and indicate this with a symbol.

**Contraindication**

- The device is not suitable for use on pregnant women and patients with implanted, electronical devices (e.g. pacemaker or defibrillator).

GB Safety instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.
- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- Consult your physician to get more information about your blood pressure. A self-diagnosis and self-treatment based on the measured values may be dangerous. Always follow the instructions given by your physician!
- If you experience discomfort during a measurement, such as pain in the upper arm or other complaints, press the START/ STOP Button **1** immediately to stop the measurement, loosen the cuff and remove it from your arm.
- The device only works correct with the fitting cuff.
- The unit is not suitable for children.
- Do not allow children to use the unit. Medical products are not toys!
- Please ensure that the unit is kept away from the reach of children.
- Do not put the air tube around your neck - danger of suffocation!
- The swallowing of small parts like packaging bag, charging cable and so on may cause suffocation.
- The user needs to check that the equipment functions properly and that it is in proper working condition before being used.
- The device may not be operated in rooms with high levels of radiation or in the vicinity of high-radiation devices, such as radio transmitters, mobile telephones or microwaves, as this could cause functional defects or incorrect measurements.
- Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquids (such as alcohol) are present.
- No modification of this unit/device is allowed.
- Please do not attempt to repair the unit yourself in the event of malfunctions, since this will void all warranty claims. Only have repairs carried out by authorized service centers.
- Protect the unit against moisture. If liquid does manage to penetrate into the unit, remove the batteries immediately and do not continue to use the unit. In this case contacts your specialist dealer or notify us directly.
- Please do not use a dilution agent, alcohol or petrol to clean the unit.
- Please do not hit the device heavily or let it fall down from a high place.
- Do not use mobile phones in the range of the device. This may lead to malfunctions.

for power supply

- The USB-C Power Supply Port **3** is used for charging the device only. The adapter used should comply with the requirement of IEC 60601-1 standard, and the specifications must meet the requirements: input: AC 100-240V 50/60 Hz, output: DC 5V 1.0A. Other AC adapters may vary in output voltage and polarity, which may pose a risk to your safety and damage the device.
- Only charge the unit's battery using the USB charging cable supplied. Never leave the unit unattended when it is connected to the mains (for charging) or in operation.
- Never charge the device in a flammable environment (e.g. flammable objects, gasoline, flammable gases, paint, etc.)
- Only connect the unit for charging so that the cable and power connector are freely accessible.
- The built-in batteries are not intended for replacement or removal by the end user. If problems occur with the power supply, contact the customer service.
- To disconnect the unit from the mains, always pull the plug out of the power connection. Do not pull on the cable!

Unit and Display

- |                                                         |                                         |
|---------------------------------------------------------|-----------------------------------------|
| <b>1</b> LED Display                                    | <b>2</b> User Selection/ Setting Button |
| <b>3</b> START/ STOP Button                             | <b>4</b> Memory Button                  |
| <b>5</b> USB-C Power Supply Port                        | <b>6</b> Date and Time                  |
| <b>7</b> Blood Pressure Indicator                       | <b>8</b> Systolic Blood Pressure Value  |
| <b>9</b> Systolic Blood Pressure Icon                   | <b>10</b> Bluetooth Icon                |
| <b>11</b> Diastolic Blood Pressure Value                | <b>12</b> Diastolic Blood Pressure Icon |
| <b>13</b> Pulse Rate Icon                               | <b>14</b> Pulse Rate Value              |
| <b>15</b> Heartbeat Icon/ Irregular Heartbeat Indicator | <b>16</b> Unit of Blood Pressure        |
| <b>17</b> Battery Indicator                             | <b>18</b> User 1 Icon                   |
| <b>19</b> User 2 Icon                                   | <b>20</b> Guest User Icon               |
| <b>21</b> Movement Detection Icon                       | <b>22</b> Memory Number                 |
| <b>23</b> Cuff Fitness Detection Icon                   | <b>24</b> Average Value Icon            |
| <b>25</b> Memory Icon                                   |                                         |

Scope of supply

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre. The following parts are included:

- 1 **medisana** blood pressure monitor
- 1 USB-C charging cable
- 1 instruction manual

Please contact your supplier if you notice any transport damage on unpacking the unit.

**WARNING - Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!**

How is blood pressure measured?

The **medisana** BU 640 connect is a blood pressure unit which is used to measure blood pressure at the upper arm. The measurement is carried out by a microprocessor, which, via a pressure sensor, measures the vibrations resulting from the inflation and deflation of the cuff over the artery.

Blood pressure classification

| systolic mmHg | diastolic mmHg | Blood pressure indicator |
|---------------|----------------|--------------------------|
| ≥ 160         | ≥ 100          | red                      |
| 140 - 159     | 90 - 99        | yellow                   |
| 90 - 139      | 60 - 89        | green                    |
| < 90          | < 60           | green                    |

**WARNING - Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!**

Influencing and evaluating readings


- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least five minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Getting started - Charging the battery

Use a 5V mains adapter with an output current of 1A. Connect the supplied USB charging cable to the adapter and to the USB-C power supply port **3** of the device (The USB-C charging cable is included in the packaging). The Battery Indicator **17** keeps flashing when charging.

Settings - Date, time and unit

In the power-off mode, press the Setting button **2** for about 4 seconds to enter the date setting interface, and the "year" will flash. Press the Memory button **4** to adjust the year, and press the Setting button **2** to confirm the selection. When the year is set, it will automatically enter the month setting. Press the Memory button **4** to adjust the month, and press the Setting button **2** to confirm the selection. Follow the same steps to adjust the date/hour/minute.

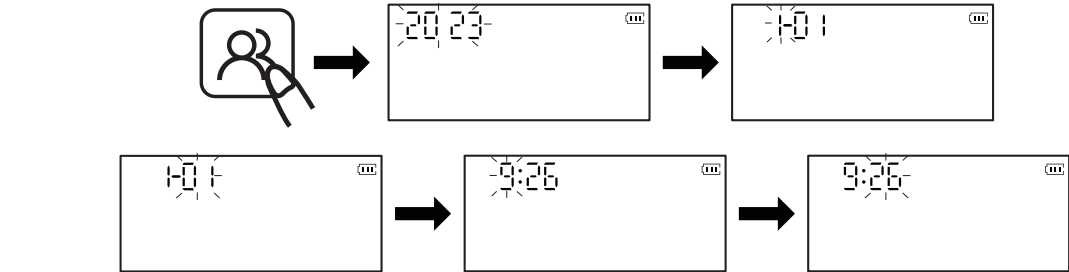In power-off mode, keep pressing the START/ STOP button **1** for about 3 seconds to enter the unit selection. Press the Memory button **4** to switch between mmHg and kPa, and then press the Setting button **2** to confirm the selection. The default unit is mmHg.

Fitting the cuff

- Slide the open end of the cuff through the metal bracket so that the Velcro fastener is on the outside and it becomes a cylindrical form (Fig.1). Slide the cuff over your left upper arm.
- Position the device's screen in the middle of your arm in line with your middle finger (Fig.2). The lower edge of the cuff should be 2 - 3 cm above the crease of the elbow. Pull the cuff tight and close the Velcro fastener.
- Tighten the end of the cuff so that it wraps evenly around your arm, leaving room for 2 fingers between the cuff and your arm.
- Measure the blood pressure on your bare arm.
- Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.
- Correct measuring position for sitting (Fig.3). Make sure that the cuff does not slip during measurement and that the printed arrow points to the crook of the arm.

User memory setting

The **medisana** blood pressure monitor features 2 separate memories, each with a capacity of 120 memory logs. In the power-off mode, press the User selection button **2** to enter the user selection interface. Then press the User selection button **2** to select between User 1 **18** / User 2 **19** / Guest User **20**. (There is no memory log for Guest User **20**).

Taking a blood pressure measurement

- Press the START/STOP button **1** to start the measurement.
- The device then automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display. Cuff Fitness Detection Icon **23** is always displayed on the screen when the cuff is worn correctly. When the cuff is worn too loosely, the icon will flash to remind you.  
The Movement Detection Icon **21** flashes when body movement is detected. It may cause inaccurate measurement results. Please restart the measurement.
- The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the heartbeat signal, the Heartbeat Icon **15** on the display starts to flash
- When the measurement is finished, the systolic and diastolic blood pressure and the pulse value appears on the display. The Blood Pressure Indicator **7** displays the relevant colored bar depending on the blood pressure classification. If the unit has detected an irregular heartbeat, the Irregular Heartbeat Indicator **15** also appears.
- The readings are automatically saved in the selected memory (1 or 2).
- After about 1 minute without further operation, the machine switches off automatically.

Transfer via Bluetooth®

This device offers the possibility to transfer your measured values via Bluetooth® to the VitaDock® app. The VitaDock® app allows the evaluation, storage and synchronisation of your data between multiple iOS- and Android-devices to have access to it anytime and anywhere. You may share your results with your friends or your doctor. Therefore you need a free user account, which you can create on the website [www.vitadock.com](http://www.vitadock.com). For Android or iOS mobile devices, you may download the respective apps. You will find a detailed instruction for how to install and use the software on the mentioned website. After each measurement an automatic transfer of the values will take place (provided that Bluetooth® is activated and configured on the receiving device).

**WARNING**

- Keep the device at least 20 centimeters away from the human body (especially the head) when the Bluetooth® data transmission is proceeding after measurement.
- The distance between the device and the receiver should be between 1 meter to 10 meters to avoid possible disturbances.

Bluetooth® type: 4.2 BLE; System requirements: Android 6.0 or newer; iOS 7.0 or newer  
RF Frequency range: 2402 MHz - 2480 MHz; Equivalent isotropic radiant power: 3,99 dBm;  
Maximum permitted power: 0,47 mW; Transmitting distance: 10 m (Class 2); Data throughput: 1 Mbps

Displaying stored results

This unit features 2 separate memories, each with a capacity of 120 memory logs. Results are automatically stored in the memory selected by the user. Press the Memory button **4** when power off, to call up the average measured values of the last 3 measurements (displayed with Average Value Icon **25**). Press the Memory button **4** again to call up the last measurement value. Repeatedly pressing the Memory button **4** to display the respective values measured previously. If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 1 minute in memory recall mode. Memory recall mode can be exited at any time by pressing the START/STOP button **1**, which will also switch the unit off. Once the memory log is full, old values will be refreshed with new ones.

Deleting stored values

If you are sure that you want to permanently remove all stored memories, press the User selection button **2** in power-off mode to select the user group whose measured values need to be deleted. Press the START/STOP button **1** to power off and then press the Memory button **4** to activate the screen. Press and hold the Memory button **4** for about 3 seconds to delete the memories of the selected user and the icon "00" will be shown on the display.

Abnormal phenomena and handling troubleshooting

| Abnormal displays | Cause                                                                           | Solution                                       |
|-------------------|---------------------------------------------------------------------------------|------------------------------------------------|
| <b>Er U</b>       | The pressure cannot reach 30 mmHg (4 kPa) in 12 seconds                         | Please follow the correct method and try again |
| <b>Er H</b>       | The inflation reaches 295mmHg                                                   |                                                |
| <b>Er 1</b>       | The pulse rate is not detected correctly                                        |                                                |
| <b>Er 2</b>       | Too much disturbance (Move, talk, or magnetic disturbance during a measurement) | Move to a closer distance                      |
| <b>Er 3</b>       | The measurement result is abnormal                                              | Please follow the correct method and try again |
| <b>Er 23</b>      | SYS value reads lower than 57mmHg                                               |                                                |
| <b>Er 24</b>      | SYS value reads higher than 255mmHg                                             |                                                |
| <b>Er 25</b>      | DIA value reads lower than 25mmHg                                               |                                                |
| <b>Er 26</b>      | DIA value reads higher than 195mmHg                                             |                                                |

Abnormal phenomena and handling

| Abnormal displays                                 | Cause                                        | Solution                                                                                          |
|---------------------------------------------------|----------------------------------------------|---------------------------------------------------------------------------------------------------|
| <b>Failure to power on</b>                        | Whether the power is insufficient            | Recharge the device with USB-C charging cable till the Battery Indicator <b>17</b> stops flashing |
| <b>No inflation</b>                               | Whether the cuff is broken or leaked         | Please contact the dealer to replace with a new cuff                                              |
| <b>Unable to measure due to the display error</b> | Whether the arm is moved when pressurization | Keep your arm and body still                                                                      |
|                                                   | Whether you talk during the measurement      | Keep quiet while measuring the blood pressure                                                     |
| <b>Air leakage of the cuff</b>                    | Whether the cuff is twined too loose         | Please tighten the cuff                                                                           |
|                                                   | The airbag of the cuff is ripped             | Please contact the dealer to replace with a new cuff                                              |

**WARNING - If the blood pressure still cannot be measured after trying the above-stated solutions, please contact the dealer. Do NOT attempt to disassemble the device by yourself.**

Cleaning and maintenance

Clean the unit using a soft cloth lightly moistened with a mild soapy solution. Never use abrasive cleaning agents, alcohol, naptha, thinner or gasoline etc. Never immerse the unit or any component in water. Be cautious not to get any moisture in the main unit. Do not wet the cuff or attempt to clean the cuff with water. Using a dry cloth, gently wipe away any excess moisture that may remain on the cuff. Lay the cuff flat in an unrolled position and allow the cuff to air dry. Do not expose the unit to direct sunlight; protect it against dirt and moisture. Do not expose the unit to extreme hot or cold temperatures. Keep the device in the storage case when not in use. Store the unit in a clean and dry location.

Disinfection

- If used by yourself in a home environment, wipe the cuff once a month with a commercially available cotton cloth with 75% alcohol for more than 1 minute at a time for disinfection in addition to the normal cleaning routine.
- If used for more than one person (home use), wipe the cuff once a week with a commercially available cotton cloth with 75% alcohol for more than 1 minute at a time for disinfection in addition to the normal cleaning routine.
- After cleaning/disinfection and before use, make sure that there are no blood stains or dirt on the display, the device or the cuff. If there are blood spots or dirt that cannot be removed, please dispose of the device without reusing it.
- If used in a complex environment (e.g. in a hospital) or by persons outside your own family, replace the cuff with a new one after use.

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

Guidelines / Standards

This blood pressure monitor meets the requirements of the EU standard for noninvasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) "CE 0123 The blood pressure monitor corresponds to European regulations EN 60601-1 and EN 60601-1-2. The specifications of EU Guideline "MDR (EU) 2017/745" are met.

Technical specifications

|                                             |                                                                          |
|---------------------------------------------|--------------------------------------------------------------------------|
| Name and model                              | : <b>medisana</b> Upper arm blood pressure monitor <b>BU 640 connect</b> |
| Display system / Memory slots               | : Digital display / 2 x 120                                              |
| Measuring method                            | : Oscillometric                                                          |
| Power supply                                | : 3.7V 450 mAh rechargeable lithium battery                              |
| Charging method                             | : USB-C charging port, charging voltage DC5V 1A                          |
| Blood pressure measuring range              | SYS: 57-255 mmHg                                                         |
|                                             | DIA: 25-195 mmHg                                                         |
| Pulse measuring range                       | : 40 - 199 beats per minute                                              |
| Maximum error tolerance for static pressure | : ± 3 mmHg                                                               |
| Maximum error tolerance for pulse rate      | : ± 5 % of the value                                                     |
| Pressure generation                         | : Automatic with pump                                                    |
| Deflation                                   | : Automatic                                                              |
| Auto power-off                              | : 1 minute after use                                                     |
| Product life span                           | : Approx. 5 years with 4 times daily use                                 |
| Operating conditions                        | : +5°C to +40°C, 15 to 90 % max. relative humidity                       |
| Storage / Transportation conditions         | : -20°C to +55°C, 10 to 93% max. relative humidity                       |
| Dimensions / Cuff                           | : Approx. 123 x 59 x 28 mm / 22 - 42 cm                                  |
| Weight (main unit)                          | : Approx. 225 g                                                          |
| Item number                                 | : 50061                                                                  |
| EAN number                                  | : 48971 38 50061 0                                                       |

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

- The warranty period for medisana products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
  - All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**Shenzhen AOJ Medical Technology Co., Ltd.**  
Room 301&4F, Block A, Building A, Jingfa Intelligent Manufacturing Park, Xiaweiyuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, China

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