

A line drawing of a right shoulder joint from a lateral perspective. The humerus is at the top, and the scapula is at the bottom. The glenoid cavity is visible. Five numbered labels point to specific features: 1 points to the acromion, 2 points to the glenoid labrum, 3 points to the glenoid cavity, 4 points to the humeral head, and 5 points to the scapula.

- ### Explanation of symbols

- Recycling symbols/codes:
These are used to provide information about the material and its proper use and recycling.



-

- 

- The device must only be used for its intended purpose as described in the instruction manual.
- Using the device for any other purpose invalidates the warranty.
- Only use the device in the proper place, that is, around the neck.

- 

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

- Use a 5V mains adapter with an output current of 2A. Connect the supplied USB charging cable to the adapter and to the USB-C charging port ③ of the device.
- When the battery level is below 15%, the white light will flash quickly, and the buzzer will beep three times to signal that charging is required.
- While charging, the white light will flash slowly; once fully charged, the white light will remain on.

- Press Power/ mode button **2** for approx. 2 seconds to switch on the device. It will default to Exercise mode, indicated by a white light and a single beep. The cervical spine will then move up and down repeatedly.
- The device will be switched off automatically after 15 minutes for the Exercise mode. Or press the Power/ mode button **2** for about 2 seconds to turn it off, with a single beep.

- Short press the Power/ mode button **2** to switch to Stationary mode, which will maintain a set elevation height; indicated by a blue light and a single beep. Press again to switch back to Exercise mode.
- The device will be switched off automatically after 60 minutes in Stationary mode. Or press Power/ mode button **2** for approx. 2 seconds to switch off; the buzzer will beep once.

- The heating function is disabled by default when the device is powered on. Short press the Heat button **4** to activate heating; the light will turn orange, followed by a beep. Press it again to turn off the heating; the light will turn off, and followed by a beep.

Symptoms	Possible Causes	Solutions
Device cannot be switched on.	1. Device is out of battery. 2. Device is switched off.	1. Please charge the device before use. 2. Press and hold the Power/ mode button 2 for 2 seconds.
Device suddenly stopped working.	1. Device is out of battery. 2. The operating time has ended. 3. Device has been used for a long time.	1. Please charge the device before use. 2. Restart the device by pressing the Power/ mode button 2 for 2 seconds. 3. Turn the device off for 1-2 hours and then restart it.
The product is loud during operation.	1.It has been used for a long period of time. 2.This is a normal sound generated by the mechanical structure during operation.	Turn the device off for 1-2 hours and then restart it.
Product Overheating	Device has been used for a long time.	Allow the product to cool for at least 1-2 hours.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your local authority or your supplier for information about disposal.

Name and model:	medisana Neck Trainer NT 105
Power supply:	Li-Ion battery 3.7V, 2000mAh Input: USB-C, 5V --- 2.0A
Max. operating time:	Exercise mode: 15 minutes; Stationary mode: 60 minutes with automatic switch-off
Operating conditions:	Only use in dry rooms at -5°C to +40°C
Storage conditions:	In a cool, dry place
Neck circumference:	30 - 49 cm
Dimensions (L×W×H):	Approx. 19.9 x 17 x 13.5 cm
Weight:	Approx. 0.45 kg ± 0.1 kg
Article No.:	50135
EAN No.:	4897138 50135 8

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for **medisana** products is one year from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

medisana®

Unit E&F, 31/F, YHC Tower, 1 Sheung Yuet Road,
Kowloon Bay, Kowloon, Hong Kong