
GB Instructions for use
Upper Arm Blood Pressure Monitor BU 509

Instruction manual - *Please read carefully!*

Unit and LC-Display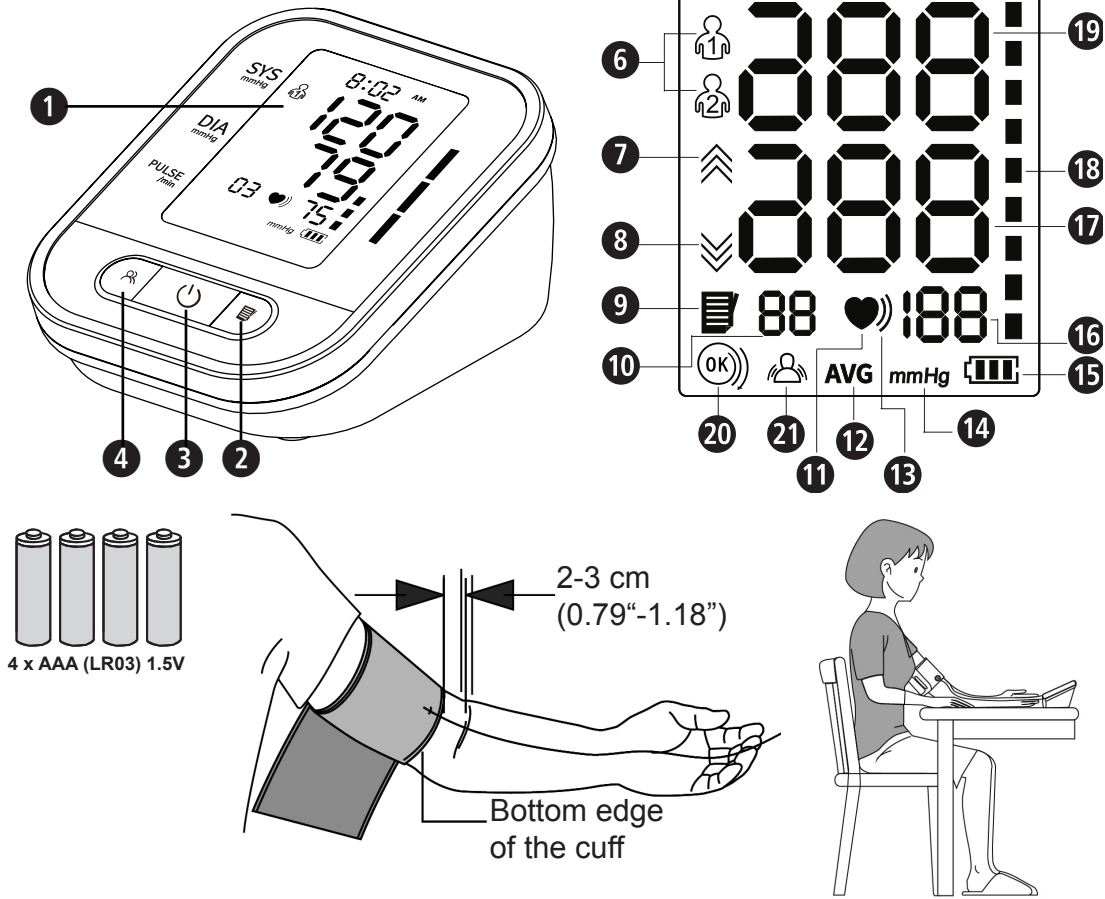

Intended use

- This upper arm blood pressure monitor is designed to measure the systolic and diastolic pressure and pulse rate of an adult person using non-invasive oscillometric technique through an inflatable cuff placed around the upper arm. The device can be used at home.

Contraindications

- There are no known contraindications.

Explanation of symbols

IMPORTANT

Follow the instructions for use!
Non-observation of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

MD Medical device

Temperature range

Ambient pressure limitation

Humidity range

Recycling symbols/codes:

These are used to provide information about the material and its proper use and recycling.

CE CE sign

This side up

Keep away from sunlight

IP21 The device is protected against splashing water. Water splashed against the enclosure from any direction shall have no harmful effects.

SN Serial number

EU REP EU authorized representative

UDI Unique device identifier

Battery change symbol

Keep dry

LOT LOT number

Manufacturer

Date of manufacture

Observe local disposal guidelines!

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- If you have any medical conditions (such as arterial occlusive disease, arrhythmia, atrial or ventricular atrial fibrillation, arteriosclerosis, poor circulation, diabetes, pre-eclampsia or kidney disease), consult your doctor before using the device. These diseases - just like movements, tremors or shivering - can lead to false readings.
- The unit may not be used to check the heart rate of a pacemaker.
- The result of a blood pressure measurement with this device is not suitable for self-diagnosis or self-treatment. To avoid health risks, always follow your doctor's instructions and contact him/her for an assessment of the measurement result.
- Never change the intake of medication prescribed by a doctor as a result of a measurement with this device and consult your doctor for advice on the best time to take the measurement before measuring.
- The device is not suitable for continuous monitoring of a patient, assessment of an existing disease or first aid.
- The device is not suitable for use on patients undergoing surgical treatment and should only be used in specified locations or in private homes.
- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- The patient is an intended operator and all the functions can be safely used.
- If discomfort such as upper arm pain, skin irritation or other discomfort occurs during a measurement, press the start/stop button **3** to deflate the cuff immediately. Loosen the cuff and remove it from the upper arm.
- This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons. Medical devices are not toys!
- Keep the appliance out of the reach of children.
- The device must not be operated in rooms with high radiation levels or in the vicinity of devices with high radiation levels such as radio transmitters, mobile phones or microwaves or in the vicinity of HF surgical devices, MRI devices (magnetic resonance imaging) or computer tomographs. This may cause malfunctions or incorrect readings.
- Protect the unit from moisture. Should liquid nevertheless penetrate the device, the batteries must be removed immediately and further applications avoided. In this case, contact your specialist dealer or inform us directly.
- Never use the unit outdoors or in damp rooms.
- Never use the unit in moving vehicles (e.g. cars).
- Do not operate the unit near flammable gas (e.g. narcotic gas, oxygen or hydrogen) or flammable liquid (e.g. alcohol).
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Do not make any modifications to the unit, do not disassemble it and never repair the unit yourself. Have repairs carried out only by authorised service centres.
- Before using the device, it is the user's responsibility to determine that the device is safe and working properly.
- Repeated blood pressure measurements in high numbers can lead to undesirable side effects, e.g. nerve crushing or blood clots.
- In case of performance fluctuations (e.g. inaccurate readings or abnormal display), stop using the device and contact customer service.
- Never place the cuff over injured skin, when a catheter is in place, or during blood transfusions.
- Talk to your doctor before using this device on an arm where there is an intravascular access or therapy or an arterio-venous (A-V) shunt, as there may be a temporary disruption of blood flow which could result in injury.
- Talk to your doctor before using the device if you have had a mastectomy or suffer from circulatory problems or blood disorders, as inflating the cuff may cause bruising.
- To avoid strangulation, always keep the cable and air tube away from children, infants and young children.
- Do not kink or bend the air hose excessively.
- Keep the device away from heavy shocks or vibrations and do not drop it.Never use force, e.g. when putting on the cuff.
- Only inflate the cuff when it is properly applied to the upper arm.
- Remove the cuff if the air is not automatically pumped out during measurement.
- Use only original accessories and spare parts from the manufacturer, otherwise damage to the unit or personal injury may result.
- Observe the local disposal guidelines if you wish to dispose of the unit.
- If you have any doubts about the proper functioning of the unit, stop using it and contact the service centre immediately.
- Swallowing small parts such as packaging material, battery, battery compartment cover, etc. can lead to suffocation. Therefore, never leave children alone with the unit.
- No maintenance or cleaning may be carried out during operation. Follow the maintenance and cleaning instructions in this manual.
- Do not measure blood pressure when other measurements are being taken on the same part of the body at the same time, as these may be disturbed or fail.
- If the ambient temperature is less than 5°C or more than 40°C, the waiting time from minimum storage temperature to use is at least 1 hour; the time from maximum storage temperature is at least 2 hours.
- Remove the batteries if you are not using the device for a long time.

SAFETY NOTES FOR BATTERIES

- Do not disassemble batteries!
- Replace all batteries if the unit display shows the low battery symbol.
- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed, seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!

- Insert the batteries correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period of 3 months or more!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Unit and LC-Display

1 LC-Display

2 User memory

3 Start/Stop

4 User selection / settings

5 Time/Date

6 User icon 1/2

7 Inflation icon

8 Deflation icon

9 Memory icon

10 Memory number

11 Pulse icon

12 Average value

13 Symbol irregular heartbeat

14 Blood pressure measuring unit

15 Battery change icon

16 Display of Pulse Rate

17 Display of Diastolic Pressure

18 Blood pressure indicator

19 Display of Systolic Pressure

20 Cuff fitness detection

21 Movement detection

Scope of delivery

First check that the unit is complete. The scope of delivery includes:

- 1 **medisana** Upper Arm Blood Pressure Monitor **BU 509**
- 4 Batteries (type AAA, LR03) 1.5V
- 1 Cuff 22 - 42 cm
- 1 Instruction manual

If you notice any transport damage when unpacking, please contact your dealer immediately.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure.

The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

Blood pressure classification	
Low blood pressure	systolic <100 diastolic <60
Normal blood pressure	systolic 100 - 139 diastolic 60 - 89
Forms of hypertension	
Mild hypertension	systolic 140 – 159 diastolic 90 – 99
Moderate hypertension	systolic 160 – 179 diastolic 100 – 109
Severe hypertension	systolic ≥ 180 diastolic ≥ 110

WARNING

Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

- Influencing and evaluating readings**
 - Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
 - Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
 - When taking readings, remember that the daily values are influenced by several factors.Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
 - Measure your blood pressure before meals.
 - Before taking readings, allow yourself at least 5-10 minutes rest.
 - If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Starting up: Insert / replace batteries

Before you can use your device, you must insert the enclosed batteries. The cover of the battery compartment is located on the back of the unit. Open it and insert 4 1.5V batteries, type AAA LR03. Pay attention to the polarity (as marked in the battery compartment). Close the battery compartment again.

Replace the batteries immediately when the battery change icon **15** appears in the display or when nothing appears in the display after the unit has been switched on.

User setting

Briefly press the “**⌂**” - button **4** to change the user setting. Pressing the “**⌂**” - button **4** briefly again switches between users 1 or 2.

Setting the date and time

The upper arm blood pressure monitor automatically stores up to 90 readings for each user (1 or 2). The memory function stores the readings with the date and time. Set the device to the current date and time before taking a measurement for the first time. The date and time cannot be changed during a measurement.

1. Press and hold button “**⌂**” **4** until the year flashes.
2. Use the button **2** to set the year and then press “**⌂**” **4** again to move to the month setting.
3. Now use the button **2** again to set the month and do the same for the day.
4. After date setting, the time needs to be set. Shortly press button **2** to switch the time system (12 or 24 hour display) and choose the time system you want.
5. After setting the time system, the computer will automatically enter the time setting mode. Adjust the hours and minutes by using the buttons **2** and “**⌂**” **4**.

How to measure correctly

- Take the measurement while sitting and preferably always on the left upper arm.
- Remove the clothing from the upper arm and relax for 5 to 10 minutes before the measurement.
- Relax your arm and rest it loosely, e.g. on a table.
- Keep calm during the measurement: do not move and do not talk, otherwise the measurement results may change.
- Take the measurement at the same time every day.
- Never measure after physical exertion or a bath - wait at least 20 to 30 minutes before taking a measurement.
- Keep your legs still during the measurement and do not cross your legs.

- Always use only the original cuff and no spare parts that have not been approved by the manufacturer.
- The following influences can noticeably change the measurement results: Measurement within one hour after a meal, consumption of alcohol, tea, coffee, restless mood or nervousness, exercise, movements (including bending forward) or drastic changes in room temperature during measurement, measurement in moving vehicles, frequently repeated measurements.

Fitting the cuff

1. Push the end piece of the air tube into the hole on the left side of the unit prior to use
2. Slide the open end of the cuff through the metal bracket so that the Velcro fastener is on the outside and it becomes a cylindrical form. Slide the cuff over your left upper arm.
3. Position the air hose in the middle of your arm in line with your middle finger. The lower edge of the cuff should be 2 - 3 cm above the crease of the elbow (see illustration). Pull the cuff tight and close the Velcro fastener.
4. Measure on the bare upper arm, the cuff should be at about heart level.
5. Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.
6. Correct measuring position in sitting position - see illustration.

Measure the blood pressure

After you have properly applied the cuff, the measurement can be started:

1. Press the START/STOP button **3** to start the measurement. Automatically, the device slowly inflates the cuff to measure your blood pressure and the display shows “00”.
2. The device inflates the cuff until a pressure sufficient for the measurement is reached. The device then slowly deflates the cuff and takes the measurement. As soon as the device detects a signal, the pulse icon **11** starts flashing in the display.
3. When the measurement is finished, the cuff is deflated. The systolic **19** and diastolic **17** blood pressure and the pulse value **16** appear in the display. According to the blood pressure classification, the blood pressure indicator **18** appears next to the corresponding coloured bar. If the device has detected an irregular pulse, the symbol for irregular heartbeat **13** also appears.
4. The measured values are automatically stored in the selected memory (1 or 2). Up to 90 measured values can be stored in each memory. If 90 values are already stored, the oldest value is overwritten.

Cuff fitness detection

The cuff fitness detection icon **20** is always displayed on the screen when the cuff is wore correctly. When the cuff is wore too loosely, the icon will flash to remind you.

Movement detection

The movement detection **21** flashes when body movement is detected. It may cause inaccurate measurement results. Please restart the measurement.

Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell) the START/STOP button **3** can be pressed at any time. The device immediately decrease the cuff pressure automatically.

Disturbing factors

If the cuff is too loose or if there is movement during the measurement, the result will not be determined correctly. Stop the procedure and repeat the measurement with the cuff correctly applied and without movements during the measurement.

Display / delete stored values

In standby mode, press the button **2** once. The unit displays the average blood pressure reading of the last three times (“AVG” **12** lights up). You can then use the button **2** to display further readings.

If you want to delete stored values, press and hold the button **2** for about 3 seconds to clear the memory. “NO” appears on the display.

Automatic switch-off

If no operation is performed for approx. 1 minute, the unit switches off automatically.

Error displays

In the event of unusual measurements, the following indications appear on the display:

Display	Cause / Solution
Er U	The inflation phase takes too long. The inflation pressure does not reach 30 mmHg within 12 seconds - please check the correct fit of the cuff.
Er H	Inflation pressure reaches 295 mmHg - automatic deflation after 20 ms.
Er 1	No pulse detected.
Er 2	Movement, talking or magnetic interference during the measurement process.
Er 3	Abnormal measurement result.
Er 23	Systolic value below 57 mmHg
Er 24	Systolic value above 255 mmHg
Er 25	Diastolic value below 25 mmHg
Er 26	Diastolic value above 195 mmHg

Troubleshooting	
Problem	Cause and solution
No display although the unit is switched on and batteries are inserted	The batteries may be empty. Insert new batteries, observing the polarity. Check whether the unit is defective. If necessary, contact the customer service.
Measurement cannot be carried out or air escapes / no pressure build-up	Put the cuff on correctly. Repeat the measurement after a 30-minute rest period. Do not talk or move during the measurement.
Each measurement produces widely differing results or a measurement is not possible because of an error display.	Repeat the measurement in the correct manner after a 30-minute rest period. Follow all the instructions given in this manual for correct measurement and common causes of incorrect measurements. Slightly deviating results are normal as blood pressure is subject to constant fluctuations.
The measured results differ greatly from the values measured by the doctor	Save the measured values and discuss them with your doctor. It is not uncommon that values not measured at home can deviate greatly.

If you cannot solve a problem, contact customer service. Do not disassemble the unit yourself.

Cleaning and care

Remove the batteries before cleaning the device. Clean the unit and the cuff with a soft cloth lightly moistened with a mild soap solution. Never use harsh detergents, alcohol, naphtha, thinner or petrol etc.. Do not immerse the unit or any accessories in water. Make sure that no moisture enters the unit. Do not use the unit again until it is completely dry. Do not expose the unit to direct sunlight, protect it from dirt and moisture. Do not expose the unit to extreme heat or cold. Clean the unit once a week. Store the unit in a clean and dry place according to the environmental conditions for storage and transport.

Disinfection

We recommend disinfecting the unit with a 70% isopropanol solution for approx. 1 minute before and after each use. To do this, proceed as follows:
Disinfect the housing with a soft cloth slightly moistened in the solution and wipe dry immediately.
Do not disinfect with high-temperature steam or ultraviolet irradiation, as this could damage the unit or accelerate its ageing!

For any cleaning and disinfection, ensure that the specified ambient conditions for operation are observed. No moisture of any kind may penetrate the unit!

Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

Guidelines and standards

This blood pressure monitor complies with the EU standard for non-invasive blood pressure monitors. It is certified according to EC directives and bears the CE mark (mark of conformity). The blood pressure monitor complies with the European regulations EN 60601-1 and EN 60601-1-2. This device is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol). The specifications of MDR (EU) 2017/745 are met.

Electromagnetic compatibility: (see attached leaflet)	
Technical specifications	
Name and model	: medisana Upper Arm Blood Pressure Monitor BU 509
Display system	: Digital display
Memory slots	: 2 x 90 for measurement data
Measuring method	: Oscillometric
Power supply	: 6V=4 x 1.5V batteries AAA LR03
Low battery	: 4.2V±0.1V (low battery voltage); <4.0V±0.1V (unit is switched off)
Blood pressure measuring range	: Systole: 57-255 mmHg Diastole: 25-195 mmHg
Measuring range pulse	: 40-199 beats/min.
Maximum measurement deviation of static pressure	: ± 3 mmHg
Maximum measurement deviation of pulse values	: ± 5 %
Mode of operation	: Continuous operation
Service life	: 5 years
Protection against electric shock	: Internally powered ME unit when using batteries, Class II ME unit when using a power supply unit.
Operating conditions	: +5 °C to +40 °C, 15 to 90 % max. relative humidity, atmospheric pressure: 70-106 kPa
Storage and transport conditions	: -20 °C to +55 °C, 10 to 93 % max. relative humidity, atmospheric pressure: 70-106 kPa. During transportation or storage, avoid shocks, direct blows, corrosive gas, exposure to sunlight or rain. Ensure good ventilation.
Dimensions (L x W x H)	: approx. 118 x 98 x 61 mm
Screen size	: approx. 50 x 65 mm
Cuff	: approx. 22 - 42 cm for adults
Weight	: approx. 230 g without batteries
Article number	: 50168
EAN number	: 4897138501686

In the course of continuous product improvements, we reserve the right to make technical and design changes.

The current version of these instructions for use can be found at www.medisana.asia

Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for medisana products is 5 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

Shenzhen AOJ Medical Technology Co., Ltd.

Room 30184-5F, Block A, Building A, Jingfa Intelligent Manufacturing Park, Xiawelyuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, PEOPLE'S REPUBLIC OF CHINA

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EU REP Share Info GmbH